
# Bike Reha

Bedieningshandleiding  
Operating Instructions  
Gebrauchsanweisung  
Mode d'emploi  
Instrucciones de manejo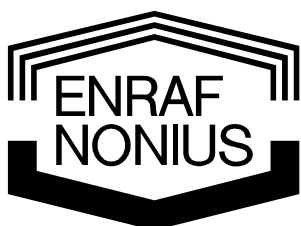

Manufacturer:

Milon Industries GmbH  
An der Laugna 2  
86494 Emersacker  
Germany

Copyright: & Exclusive Sales and Service

Enraf-Nonius B.V.  
P.O. Box 12080  
3004 GB Rotterdam  
The Netherlands  
Tel.: +31-(0)10 – 20 30 600  
Fax: +31-(0)10 – 20 30 692  
[info@enraf-nonius.nl](mailto:info@enraf-nonius.nl)  
[www.enraf-nonius.com](http://www.enraf-nonius.com)  
Part number: 1413754-44

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## TABLE OF CONTENTS

<b>1</b>	<b>Foreword</b> .....	<b>29</b>
<b>2</b>	<b>Set-up</b> .....	<b>29</b>
	2.1 Placement.....	29
	2.2 Electrical connection.....	29
	2.3 Initial use.....	29
	2.4 Environmental information.....	29
<b>3</b>	<b>Safety instructions</b> .....	<b>30</b>
	3.1 Safety aspects.....	30
	3.2 Legal aspects.....	30
	3.2.1 Warnings.....	30
	3.2.2 Caution.....	30
	3.2.3 Note.....	31
<b>4</b>	<b>General</b> .....	<b>32</b>
	4.1 Parts list.....	32
	4.2 Control panel.....	33
	4.3 LCD screen.....	33
	4.4 Menus.....	33
	4.5 Heart rate monitoring.....	33
	4.5.1 Ear sensor.....	34
	4.5.2 Chest strap.....	34
	4.5.3 Hand/pulse measurement.....	35
	4.5.4 Maintenance of the hand/pulse sensors.....	35
	4.6 Adjusting the Bike Reha.....	35
	4.6.1 Steering.....	35
	4.6.2 Saddle.....	35
	4.6.3 Adjustable cranks.....	35
<b>5</b>	<b>Contra-indications</b> .....	<b>36</b>
	5.1 Absolute contra-indications:.....	36
	5.2 Relative contra-indications:.....	36
<b>6</b>	<b>Intended use</b> .....	<b>36</b>
<b>7</b>	<b>Operation in general</b> .....	<b>37</b>
	7.1 Manual operation.....	37
	7.1.1 Constant power (W).....	37
	7.1.2 Constant torque (Nm).....	37
	7.1.3 Constant heart rate (bpm).....	37
<b>8</b>	<b>Quick-start</b> .....	<b>38</b>
	8.1 Quick-start in manual operation.....	38
<b>9</b>	<b>Manual operation</b> .....	<b>39</b>
	9.1 Configuring a profile.....	39
<b>10</b>	<b>Results</b> .....	<b>41</b>
<b>11</b>	<b>Changing the preferences</b> .....	<b>42</b>
<b>12</b>	<b>Using the Bike Reha with EN-Track software</b> .....	<b>44</b>
	12.1 General.....	44
	12.2 Possibilities with EN-Track.....	44
<b>13</b>	<b>Product liability</b> .....	<b>46</b>
<b>14</b>	<b>Maintenance</b> .....	<b>46</b>
<b>15</b>	<b>Conclusion</b> .....	<b>46</b>
<b>16</b>	<b>Technical details</b> .....	<b>47</b>
<b>17</b>	<b>Ordering details</b> .....	<b>48</b>

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## 1 Foreword

---

Thank you for purchasing the Bike Reha. This ergometer will bring great flexibility to your cardiovascular training routines.

The packaging contains 1) the Bike Reha, 2) these Operating Instructions.

In general, six of our products can be used for cardiovascular training. In addition to the Bike Reha (fig. 1, page 30), these include the Step, the Stair, the Crosswalker, the Cruiser and the En-Mill treadmill.

When designing this ergometer a great deal of work went into ensuring that it was reliable, safe, user-friendly and durable. To ensure that you use your Bike Reha properly and safely it is important that you read through the following instructions *before* you start using it.

---

## 2 Set-up

---

### 2.1 Placement

If the floor is uneven then the height of the two rubber feet on the crossbars can be adjusted to compensate. These feet can easily be rotated and fixed into position using suitable tools.

### 2.2 Electrical connection

To supply power to the electronics a protective earthing 230V socket is required. The device is thermostatically protected to prevent overheating. The maximum power consumption is approximately 40VA.

### 2.3 Initial use

After placing the Bike Reha in position you must wait for several hours before you can use it for the first time! This is necessary because of possible condensation caused by large differences between internal and external temperatures.

The connection for the power cord will be found on the left-hand side of the rear of the frame (see fig. 1, page 30) After the power cord has been connected, move the switch to position 1. The Bike Reha is now ready for use. To further operate the device, use the membrane keyboard. The various operations to be performed on the keyboard are explained in these Instructions.

### 2.4 Environmental information

During the use of the Bike Reha no emissions appear that are outside the permissible range and that will cause any environmental burden.

Your Bike Reha contains no materials that are noxious for the environment. As thus, at the end of the duration of life, no special concern is needed. However, specialised companies can take apart the Bike Reha and sort out the materials for recycling. By doing so you contribute to a better environment.

Please ensure that you are well informed of the local rules and regulations regarding to the removal of equipment and accessories.

## 3 Safety instructions

### 3.1 Safety aspects

Safe execution of the procedures in this manual requires technical experience and a general knowledge of safety precautions as well as a general knowledge of the human loco-motor system.

Warnings, Cautions and Notes have been used throughout the manual to bring special matters to the reader's immediate attention.

- A **Warning** concerns danger to the safety of the user or technician;
- A **Caution** concerns an act that might damage the equipment;
- A **Note** highlights a statement that deserves more emphasis than the general text, but which does not merit a **Warning** or a **Caution**.

It is important to follow the correct sequence of steps in a procedure to ensure safe handling of the equipment and the prevention of damage. Never, therefore, change the sequence of a procedure.

### 3.2 Legal aspects

This manual and the information it contains are the property of Enraf-Nonius B.V. (The Netherlands.) Enraf-Nonius B.V. disclaims any responsibility for personal injury and/or damage to equipment caused by:

- Negligence or disregard of a Warning or a Caution.
- Deviation from any of the described procedures.
- Performance of activities that are not described in this manual.
- Ignorance of the general safety precautions for handling tools and the use of electricity.
- Usage of the device in a manner other than that specified in this manual.

#### 3.2.1 Warnings

- All the cardiovascular training equipment is intended for use under (para)medical supervision.
- The operator shall take notice of the physical and mental condition of the patient and, to his judgement, decide whether exercising is admissible.
- Stop the training **immediately** if the patient feels unwell or dizzy, or if he/she experiences cardiac pathology symptoms!
- Check that your patients are complying with the pre-set maximum heart rate limit, and stop the training immediately if this limit is exceeded.
- When using the Polar® chest strap, a sufficient distance (at least 1.5 metres) must be allowed between users. If not, there is a risk that the pulse measurement will be distorted.
- In case of a dermatological condition, injured skin shall never contact the keyboard, heart-rate sensor, ear sensor, hand sensor, saddle, handlebar or pedals.
- Be assured that all adjustable items (levers etc.) are properly adjusted with respect to each other and well tightened. Note, danger of injury!

#### 3.2.2 Caution:

- A check shall be carried out on the packaging and on the device and for deterioration, before putting the Bike Reha into use.
- The Bike Reha must be connected to a 230V/50Hz outlet. If the wrong voltage is used then the Bike can be damaged or may function incorrectly.
- The Bike Reha shall not be installed in a location near a heat source. Exposure to direct sunlight shall be avoided.
- The device shall not be used in close proximity (i.e. less than 5 metres) of short-wave and microwave equipment.
- Never insert your hand under the cover! Through the open underside you can come into contact with the pedal mechanism, the belt or the brake. There is a **risk of injury** when the pedal shaft is rotating!
- When **any** maintenance or service work is being performed and whenever the side-cover of the housing is removed the Bike Reha **must be disconnected from the mains!**

- An uneven surface can adversely affect the generally very high level of **stability** of the Bike Reha. Correct adjustment of the rubber feet can **guarantee** the Bike's stability.
- The Bike Reha is a specialised device for blood circulation and ergometer training and is not an 'exercise bike'! Inexpert use can damage the Bike Reha.
- The patient shall receive instructions for correct operation of the Bike Reha.
- Using the Bike Reha in a manner other than that described in these Instructions can lead to hazardous situations. You are advised to read through these Instructions carefully before starting to use the Bike Reha.
- If the power cord gets damaged, de-energise the device immediately using the switch (9). Next, remove the power cord from the socket to prevent injury.
- If general faults, fluctuations in resistance or faults in the control panel occur then stop the training immediately and switch off the device.
- Make sure that the various adjustable parts (saddle, handlebar) are firm and tight.
- Keep unsupervised children away from the training equipment at all times.
- The user must assume a correct biomechanical position.
- Training incorrectly can lead to injury. Supervision by an expert (para)medic is therefore essential.
- The Bike Reha shall not be used for diagnostic purposes
- The apparatus may not be used in so-called "wet rooms" (hydrotherapy rooms)
- The apparatus should only be used by patient within the permissible weight range as indicated in the technical specifications
- The heartrate monitoring system with earclip or hand/wrist measurement is only intended for heartrate measurement. For heartrate targeted training only the **Polar®** system with chestbelt is allowed
- A daily pre-use check shall be carried out on:
  - the heart rate measuring sensors
  - the moving parts
  - the seating position adjustment parts
  - handlebar adjustment parts
  - the keyboard

Note: all parts that come in contact with the patients should be cleaned and disinfected by means of a 70% alcohol solution (EEC No. 603-000-00-5)

### 3.2.3 Note:

- If any faults should arise in your Bike Reha, please contact your equipment supplier or Enraf-Nonius B.V. (The Netherlands). Any repairs and installations should only be performed by a service technician who has been authorised by Enraf-Nonius.
- Repairs performed by service technicians who are not authorised by Enraf-Nonius and/or the use of non-original (replacement) parts and/or auxiliary equipment invalidates the warranty and can be hazardous.
- An annual check is necessary. Technical maintenance shall only be done by personnel, authorized by Enraf-Nonius. A maintenance checklist is available in the service manual.

## 4 General

The Bike Reha can be used for various purposes, ranging from fitness to rehabilitation. The Bike Reha is menu-driven. Settings and ergometer parameters can be defined via the control panel that is integrated into the steering mechanism of the Bike Reha.

Fig. 1. Bike Reha

### 4.1 Parts list

1. Connector for power cord
2. Bike Reha rear support
3. Saddle height adjustment
4. Horizontal adjustment - saddle
5. Horizontal adjustment - steering
6. Vertical adjustment - steering
7. Control panel
8. Pedals (with foot straps)
9. On/off switch

## 4.2 Control panel

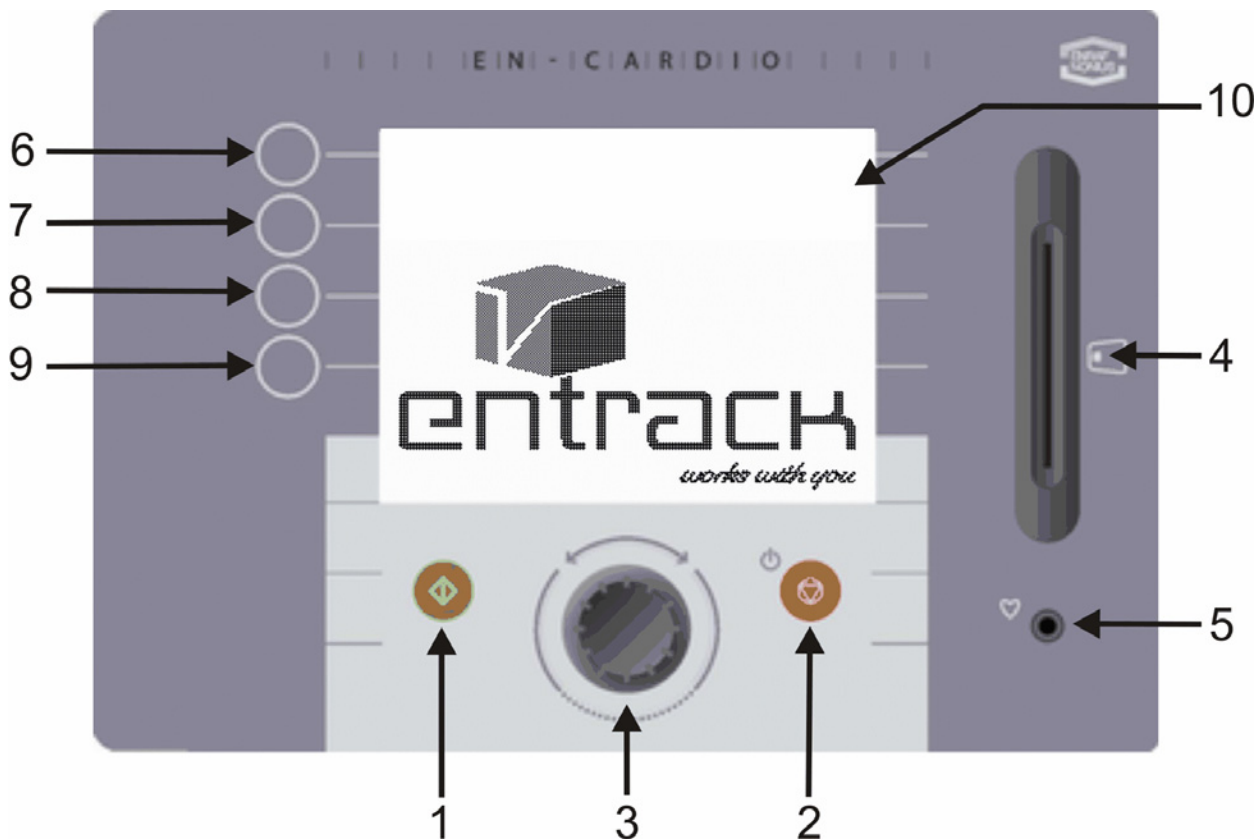
Fig. 2. Control panel

The control panel of the Bike Reha consists of the following: (Fig. 2)

11. **Start** switch to turn the Bike Reha on
12. **Stop** switch to turn the Bike Reha off
13. Rotary button for various adjustments
14. Chipcard reader
15. Ear clip for heart rate receiver
16. Selection switch 1
17. Selection switch 2
18. Selection switch 3
19. Selection switch 4
20. LCD screen

## 4.3 LCD screen

The control panel (Fig. 2) contains an LCD screen. During the preparations for a training session the control panel displays the menu. During the session itself it shows the various parameters and a graphical representation of the session.

## 4.4 Menus

The Bike Reha is menu-driven. In the menu you can select various training modes. The Bike Reha can be operated with the switches and the rotary button on the control panel. You will find a detailed explanation of the various menus in chapters 5 to 10 of these Instructions.

## 4.5 Heart rate monitoring

There are 3 ways of monitoring the heart rate:

1. ear sensor
2. chest strap (option)
3. hand/pulse measurement (option)

#### 4.5.1 Ear sensor

Various ear sensors that exist on the market can be connected to the Bike Reha.

However, the Enraf-Nonius program does not include ear sensors since we can not guarantee proper functioning of such sensors when measuring heart rate frequency.

For accurate heart rate measurement we strongly advise our users to make use of the Polar® chest belt (optional accessory).

Use an ear sensor to measure your heart rate as follows:

- Attach the ear sensor to your ear lobe.
- Attach the collar clip to your collar or shirt. This stops the sensor from moving or being disturbed during use. The wire of the ear sensor must be sufficiently long.
- The sensor can be attached to the ear lobe or (even better) can be inserted into the shell of the ear.

**After usage, clean the sensor according to the instructions of the manufacturer.**

**For most sensors a mild soap solution is sufficient. Do not use detergents!**

#### 4.5.2 Chest strap

While training on the Bike Reha, a Polar® heart rate monitoring system can also be used. The Polar® heart rate monitoring system consists of a lightweight transmitter strap worn round the chest and a receiver which is inserted into the control panel of the Bike Reha. The transmitter uses two electrodes to record each heartbeat and then sends the signals to the receiver. During a training session the heart rate appears in the upper right-hand corner of the screen.

Use the chest strap to measure your heart rate as follows:

1. Attach the transmitter to the elastic band

2. The chest strap must be comfortable and must fit the user's chest snugly

3. Moisten the grooved rear surface of the transmitter, where the electrodes are located

4. Check that the part with the transmitter is positioned correctly, i.e. on the right-hand side with the Polar logo in the middle

**Disconnect the ear sensor from the control panel. If you do not do so then there will be no telemetric monitoring via the chest strap.**

The chest strap works best if the transmitter is worn against the bare skin. If you prefer to wear the transmitter over a shirt then you must also moisten the part of transmitter the where the grooved surfaces rest against the shirt.

Maintenance of the chest strap:

- Clean the transmitter after use with a mild soap solution
- Rinse with clean water
- Dry the transmitter thoroughly, otherwise the internal batteries will run down

### 4.5.3 Hand/pulse measurement

During training on the Bike Reha, a Polar® hand/pulse monitoring system can also be used (optional). First disconnect the ear sensor from the control panel. Grasp the two hand/pulse sensors that are attached to the steering column firmly in both hands. When a pulse value is measured this will appear in the top right-hand corner of the control panel. It can take some time (about 20 seconds) before the measured pulse value appears on the screen. Throughout the training session keep your hands on the two hand/pulse sensors.

### 4.5.4 Maintenance of the hand/pulse sensors:

- Clean the sensors carefully after use with a mild soap solution
- Rinse with clean water
- Dry the sensors again thoroughly

## 4.6 Adjusting the Bike Reha

On the Bike Reha you can make the following adjustments to optimally match the Bike Reha to your patient.

### 4.6.1 Steering

To adjust the steering in the vertical dimension, turn the two handles (Fig. 1, number 6) on the front of the Bike Reha to the left. You can then adjust the steering column to the desired height and tighten the handles again by turning them to the right.

To adjust the steering in the horizontal dimension, turn the two handles (Fig. 1, number 5) to the left. You can then adjust the steering in the desired direction and tighten the handles again by turning them to the right.

These handles should not be used to move the Bike Reha as this may damage them.

### 4.6.2 Saddle

The saddle can be adjusted both vertically and horizontally. Pull the knob (Fig. 1, number 3) towards you while adjusting the height of the saddle. Make sure that the knob is inserted into the hole at the desired height. The saddle can also be adjusted horizontally (Fig. 1, number 4). Turn the handle to the left and then adjust the saddle to the desired position. After the adjustment, tighten the handle by turning it to the right again.

The saddle height of the Bike Reha is at its optimum when the client's heels are resting on the lowest pedal position while his or her leg is fully extended.

### 4.6.3 Adjustable cranks

Optionally the Bike Reha is also available ex-works with adjustable cranks. These adjustable cranks allow for optimum training even when the client is restricted in his or her movements. No tools are required to adjust the cranks.

To guarantee a correct and safe training technique the cranks must be adjusted by the trainer/physiotherapist.

---

## 5 *Contra-indications*

---

### 5.1 **Absolute contra-indications:**

- Myocardial problems in an unstable phase
- HNP in the (sub-)acute stage (with radiation and neurological abnormalities)
- Severe osteoporosis
- Spontaneous fractures
- Severe mechanical instability
- Early postoperative joint modifications

### 5.2 **Relative contra-indications:**

- Myocardial problems in a stable phase
- HNP in the subacute stage (without radiation and neurological abnormalities)
- Mild osteoporosis
- Postoperative joint modifications
- Tumours
- Inflammations in the area to be activated
- Undesirable articular load in the area to be activated (e.g. in M. Osgood-Schlatter, bursites, etc.)
- Undesirable ligamentous load in the area to be activated (e.g. overstretching with swelling)
- Undesirable soft tissue loading in the area to be activated (e.g. in very severe atrophy)
- Cardiovascular limitations
- Pulmonary limitations
- Psychogenic limitations
- Pain of such a nature that movement does not promote recovery

---

## 6 *Intended use*

---

The Bike Reha is an ergometer for medical training purposes.

As thus the Bike Reha can be used for:

- Prevention: regular use should prevent cardio-respiratory and locomotor impairments
- Rehabilitation: muscular dysbalances can be adjusted / equalized by means of regular use

## 7 Operation in general

This chapter describes how the Bike Reha is used **without** the chipcard function. All the machines in the EN-Cardio group have a uniform operation. For a description of how EN-Track is used **with** the chipcard function, please consult chapter 10.

The Bike Reha must be operated under the supervision of the physiotherapist or other suitably qualified person. Only the therapist should attempt to interpret the results of the training session.

### 7.1 Manual operation

The Bike Reha can be operated manually. All you need to do is decide which form of exercise you want to execute.

The options are:

- Constant power
- Constant torque
- Constant heart rate

#### 7.1.1 Constant power (W)

This position ensures that the power remains constant irrespective of the step speed and heart rate: the power (20-400W) can be increased or reduced using the rotary button (even during the training) in 5W increments.

#### 7.1.2 Constant torque (Nm)

The resistance will be maintained at a constant level so that a constant torque (constant resistance) is created. The value of a constant torque (5-60 Nm) can be increased or reduced using the rotary button (even during the training) in 0.5 Nm increments.

#### 7.1.3 Constant heart rate (bpm)

The ergometer will bring the patient's heart rate to the pre-set value and then keep it there. This is done by adjusting the power so that the pre-set heart rate (according to the signal from the Polar® chest strap, ear clip or the hand/pulse measurement) is achieved and maintained. The measured value is shown at the extreme right of the first line of the exercise screen, under the heart indicator. The desired value can be increased or reduced in 1 bpm increments (between 40 - 210 bpm).

During the training (in which a warm-up is recommended so that the heart rate does not rise too quickly) the pre-set desired heart rate can be changed by using the rotary button to set the new value. After the rotary button has been operated the desired value will also appear at the bottom-centre of the control panel.

## 8 Quick-start

### 8.1 Quick-start in manual operation

With the quick-start function you can start training immediately. Proceed as follows:

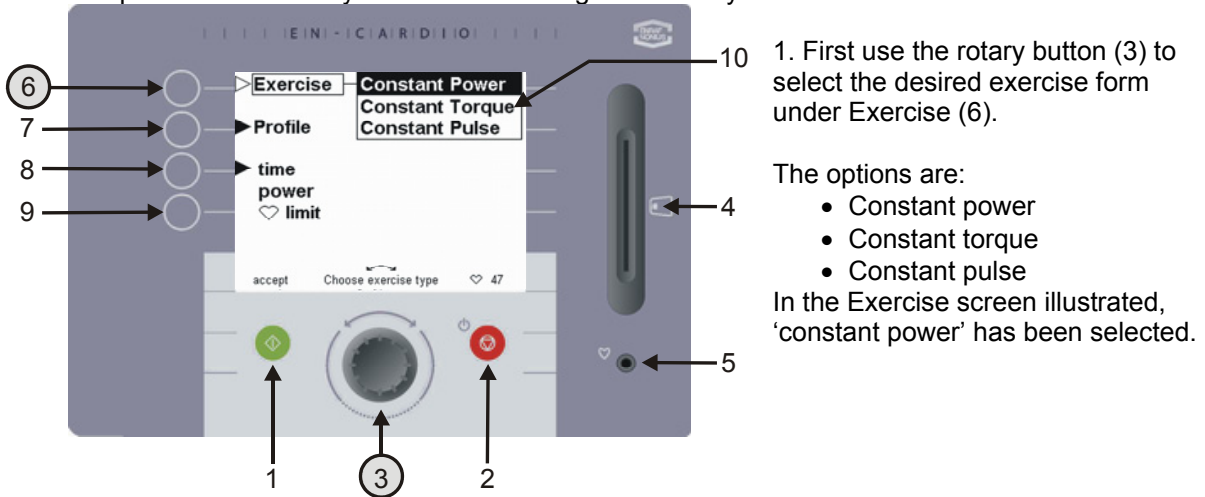
Fig. 3. Exercise options

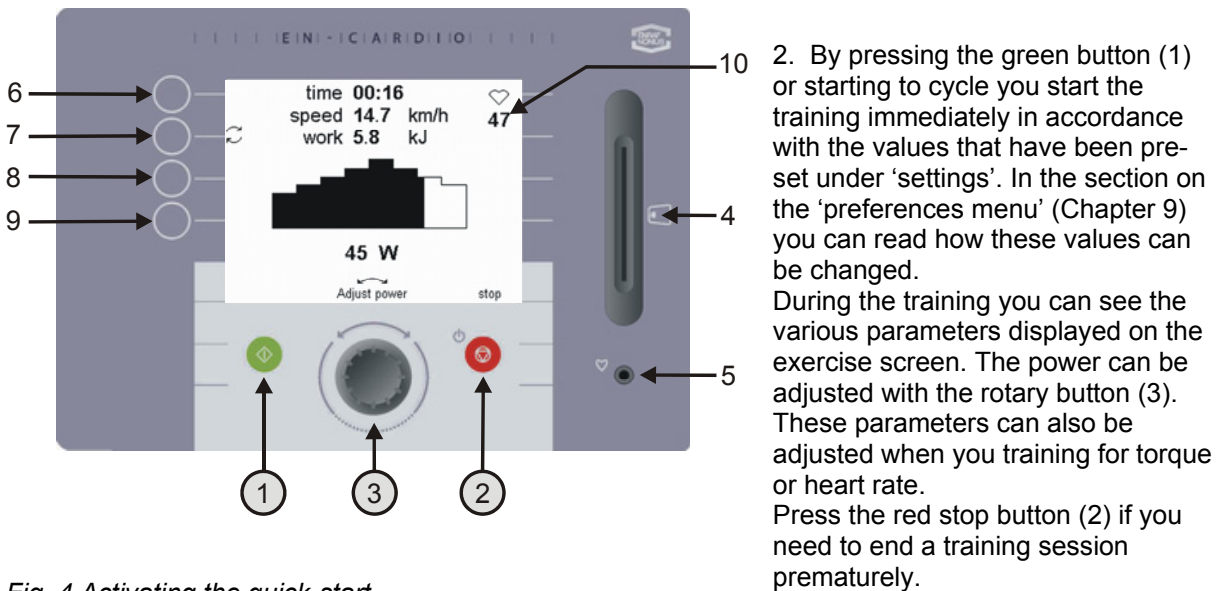
Fig. 4 Activating the quick-start

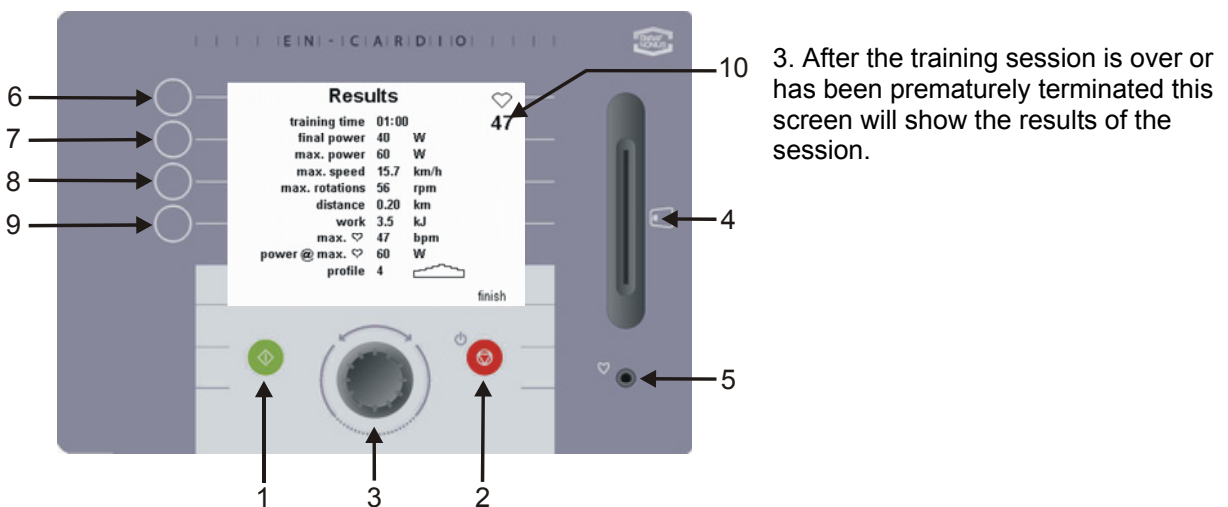
Fig. 5. Training results

## 9 Manual operation

### 9.1 Configuring a profile

On the Bike Reha you can pre-set constant power, constant torque or constant heart rate profiles. Below the procedure for configuring a constant power profile is discussed.

If you have selected a constant torque or a constant heart rate profile then you should follow the same steps as described above.

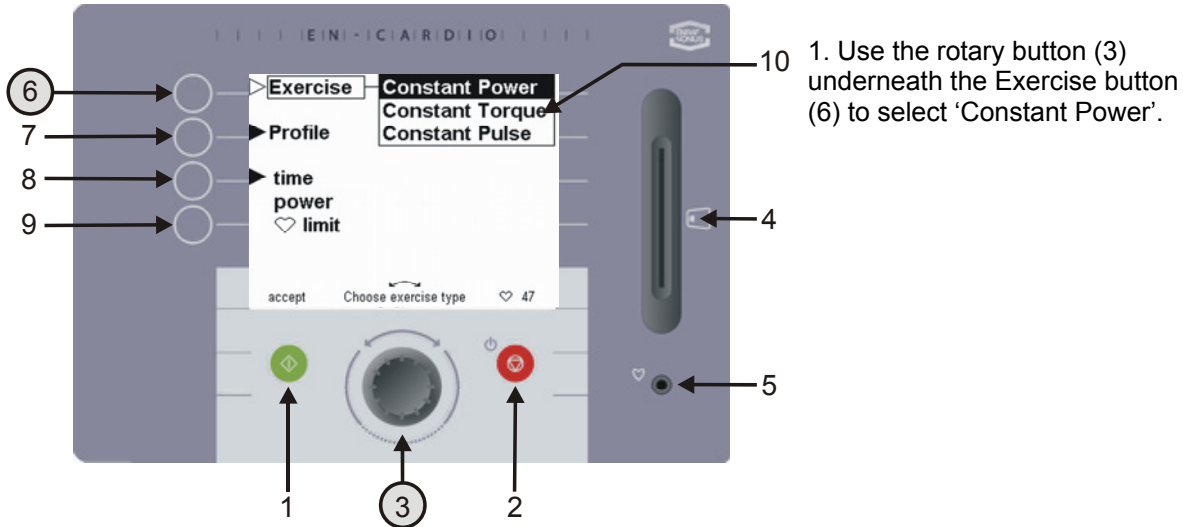
Fig. 6. Training options

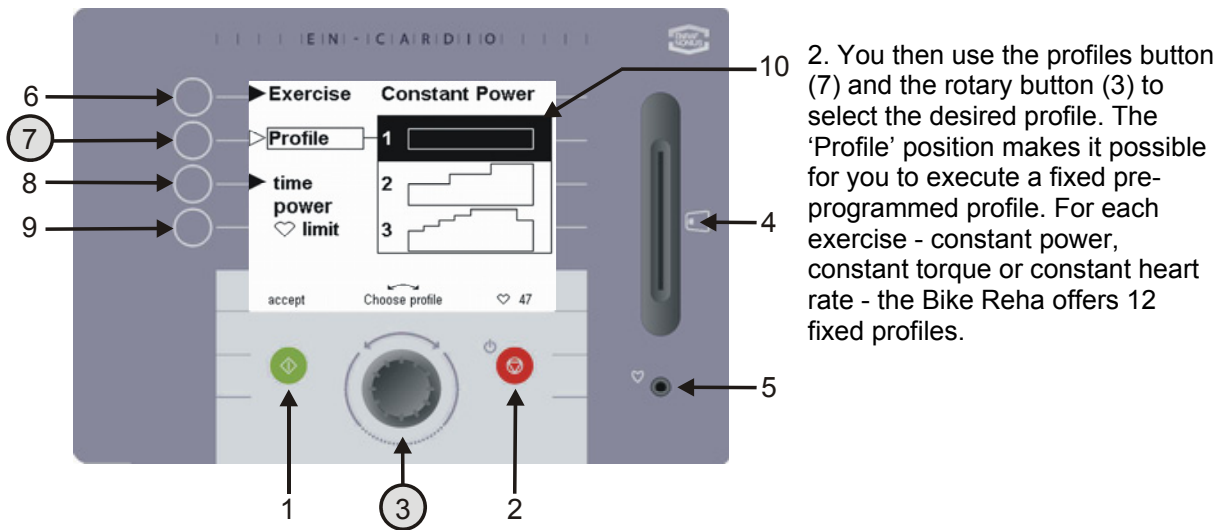
Fig. 7. Profile selection

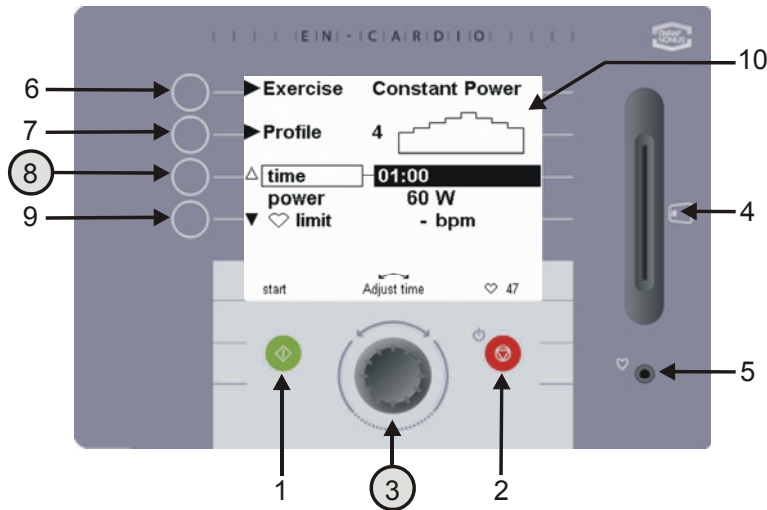
Fig. 8 Entering the training time

3. By pressing the Time button (8) you can set the duration of the exercise. The value for the time (between 00:30 seconds and 8:00 hours) can be changed with the rotary button (3). If a time of 0:00 is set then the training time will not expire but will instead run until a maximum time of 08:00 hours is reached.

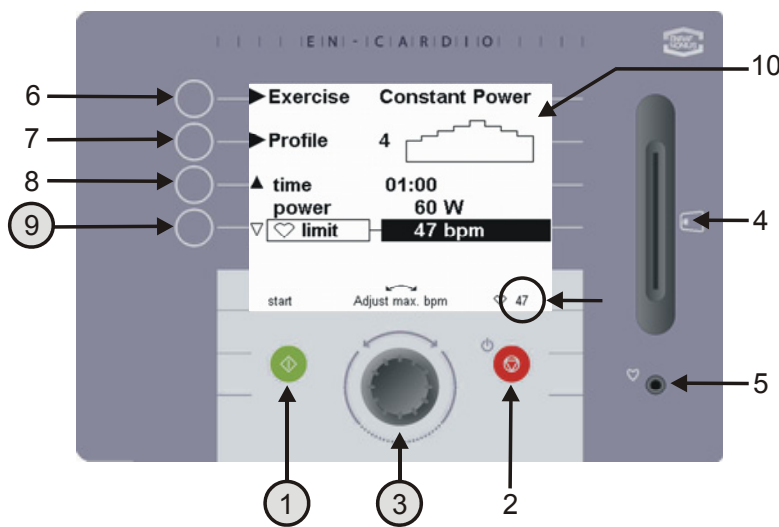
Fig. 9 Entering the resistance (W)

4. By pressing button 9 you can use the rotary button to set the maximum value of the selected profile. The starting value of this profile is half this value! If the pre-set value is 60 W, then the profile will start at 30 W. Allowance must be made for this when executing a constant heart rate profile. A gradual build-up of a heart rate profile is desirable, so make due allowance for this when selecting the various profiles!

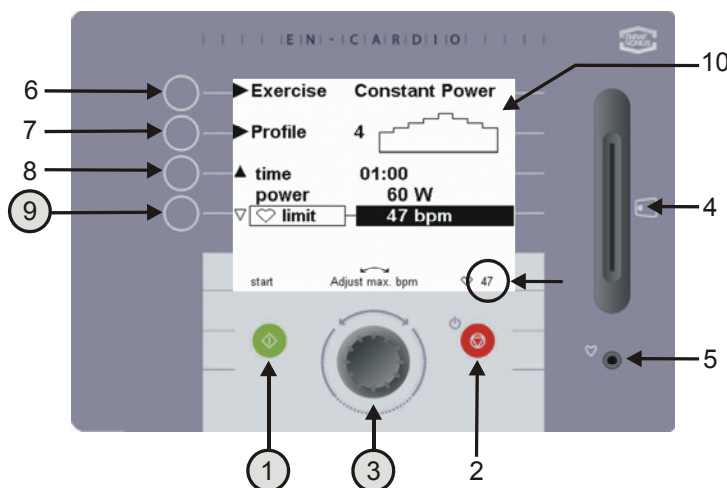
Fig. 10 Entering the maximum heart rate

5. By pressing button 9 once again you will be able to use the rotary button (3) to pre-set a maximum heart rate limit (between 0 and 255 bpm) for the training session. This acts a maximum heart rate warning mechanism.

*In other words: if you are training for resistance or torque you can pre-set a warning for the maximum heart rate.*

There will be an acoustic and an optical warning if this maximum heart rate is exceeded. The resistance must be reduced by the user **himself** using the rotary button (3). The resistance is **not** adjusted by the Bike Reha, as is the case with the Constant Heart rate exercise form. By pressing the green button (1) you start the training, and the exercise screen appears.

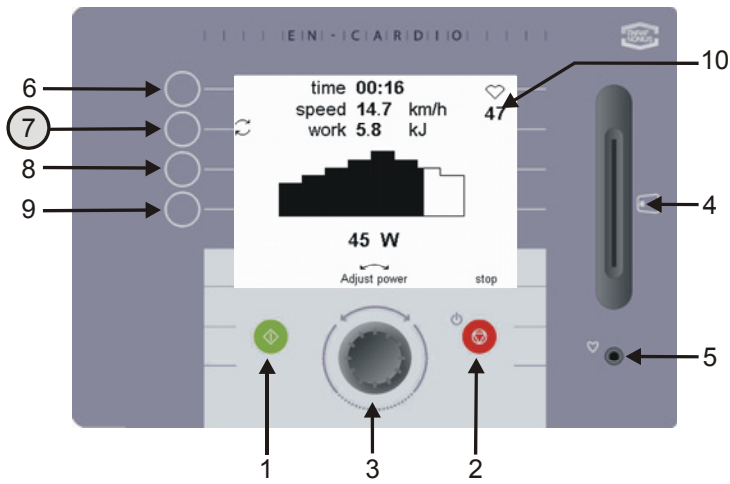
Fig. 11 Changing the value displayed in the exercise screen

6. During the training session, seven different effort values can be displayed.

The following three values are continuously displayed:

**Time:** Training time remaining  
**Speed:** Speed in [km/h] (this depends on the pre-set power in W)

**Heart Rate:** The heart rate is continuously displayed in Beats Per Minute [bpm] at the top-right of the LCD screen, under the ♥ symbol.

The following four values are shown alternately when button (7) is pressed.

The possible values are:

**Distance:** Distance covered in [km]

**Energy** : Effort produced in [kJ]

**Revs:** Pedal revolutions in revs per minute [rpm]

**Power:** Power, shown in [W]

## 10 Results

When the training time has elapsed or if you have pressed the stop button (2) during the training, this overview screen will appear containing the results of the training just performed.

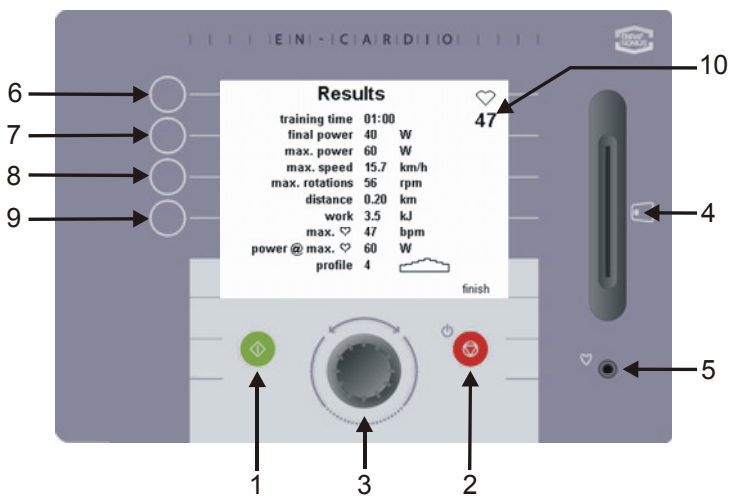
Fig. 12 Overview screen

The results displayed are:

- Training time .....01:00
- Final power.....40 W
- max. power .....60 W
- max. speed .....15,7 km/h
- max. rotations .....31 rpm
- distance..... 0.20 km
- work.....11,3 kJ
- max. ♥.....111 bpm
- power @ max. ♥ .....55 W (@ = at the moment of the max. heart beat)
- profile .....number

## 11 Changing the preferences

You can use the 'changing preferences' function to change the manufacturer's settings. These changes can only be made by the therapist or coach. To access the preferences menu you need to perform a special action (this is to prevent changes being made by unauthorised persons).

If you want to change the manufacturer's (preference) settings you can do this in this menu.

An example of manufacturer's settings is the quick-start option (chapter 6)

The manufacturer's settings are as follows (see Fig. 14):

- 'Constant Power' exercise
- Profile 1
- Training time 5:00 min
- Etc.

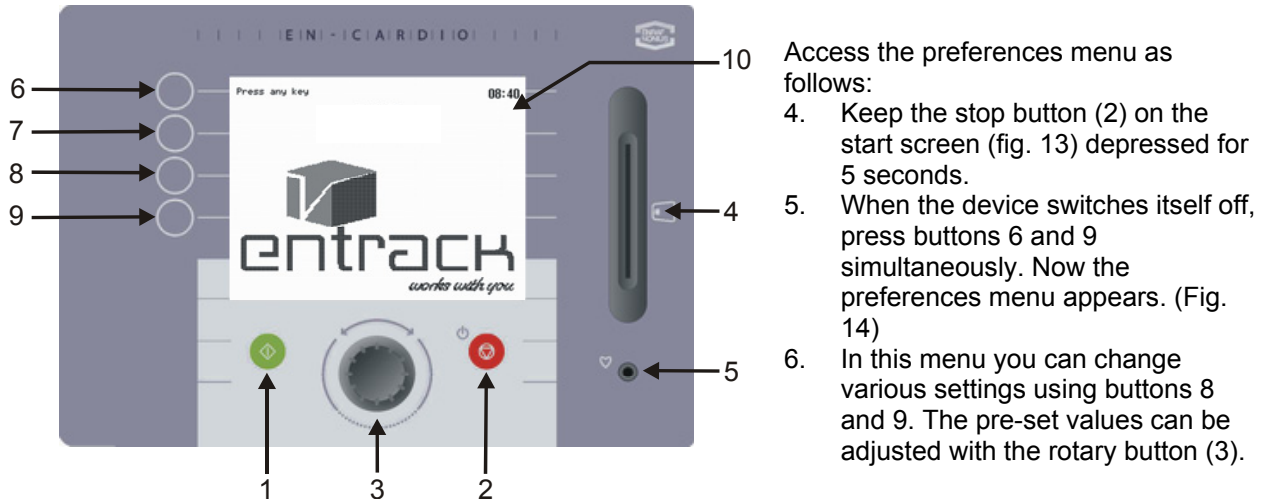
Fig. 13 Changing settings from the stand-by screen


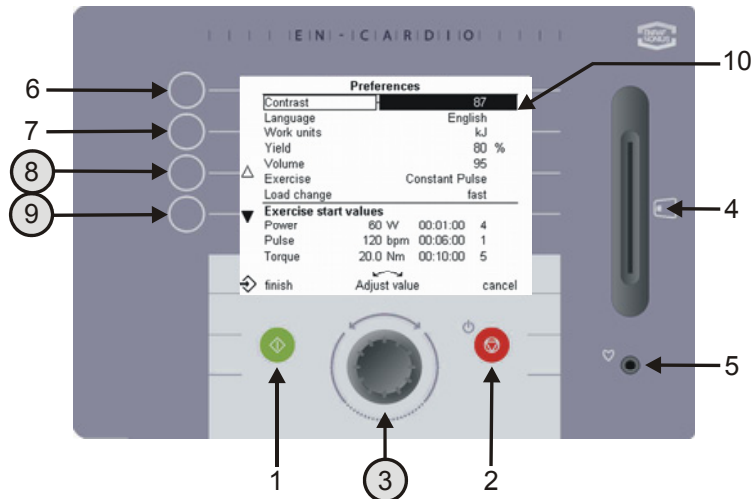
Fig. 14 Settings screen after entering the preference

The following settings can be changed with the rotary button (3):

- Contrast Set the display screen clarity
- Language Choose your own language
- Energy unit Choose between calories and joules
- Efficiency Energy burned by the client = pre-set power of the Bike Reha x efficiency %. The average value is 20%.
- Volume Set the desired volume
- Exercise Specify the form of exercise that begins with a quick-start
- Resistance change The transition in resistance can occur slowly, at average speed or quickly
- Year, month, day Enter the present date
- Time Enter the time
- Switch-off time Enter the time (5min - 60min), after which the machine will switch off
- Exercise start values Define the 'default settings' for the forms of exercise 'constant torque', constant power and constant heart rate for the following values:
  - resistance (W, Nm and bpm)
  - training time
  - profile number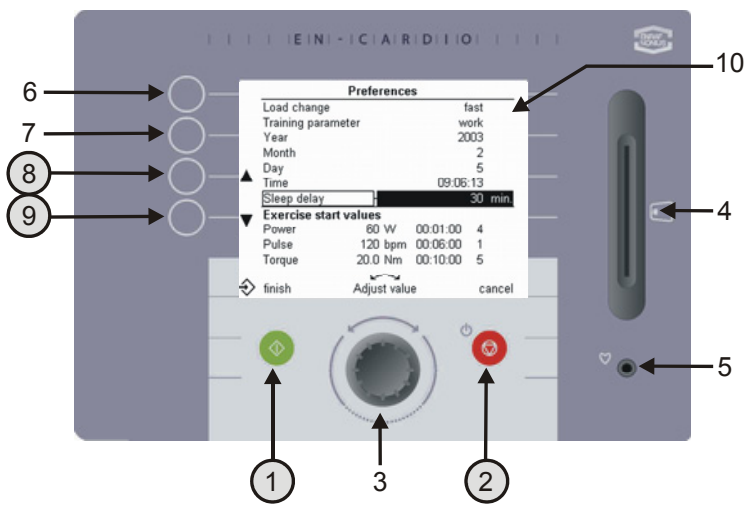

After making the changes you can exit the preferences menu by pressing the green exit button (1). The selected form of exercise, training time and profile number will then appear in the settings screen (Fig. 15). If you want to exit the screen **without** saving the changes, press the red break button (2). The settings screen (Fig. 15) appears.

Fig. 15 Preference settings

## 12 Using the Bike Reha with EN-Track software

### 12.1 General

The Bike Reha is an addition to the existing range of EN-Strength exercise machines. This part of the documentation describes the use of the Bike Reha with the Track System, and gives some examples to make it easier to understand. To use the Bike with the Track System you need to have a mastery of the EN-Track software, because controlling the Bike Reha is very similar to controlling the EN-Strength exercise machines. If you are unsure, consult the software help for EN-Track or the EN-Track software manual.

### 12.2 Possibilities with EN-Track

Using EN-Track enables you to execute more profiles on the Bike Reha. The number of profiles corresponds to the number of profiles you can create with the EN-Track system.

The Bike Reha can be used in an exercise profile in a random sequence, just like the EN-Strength, EN-Free and EN-TreeP equipment.

To create a controlled training, you can select from a multiplicity of manually created training profiles. You can enter these on the chipcard via just a few actions.

To create an exercise in the EN-Track software, select the tab EN-Cardio/Bike Reha exercises. For further explanation, please refer to the EN-Track software manual.

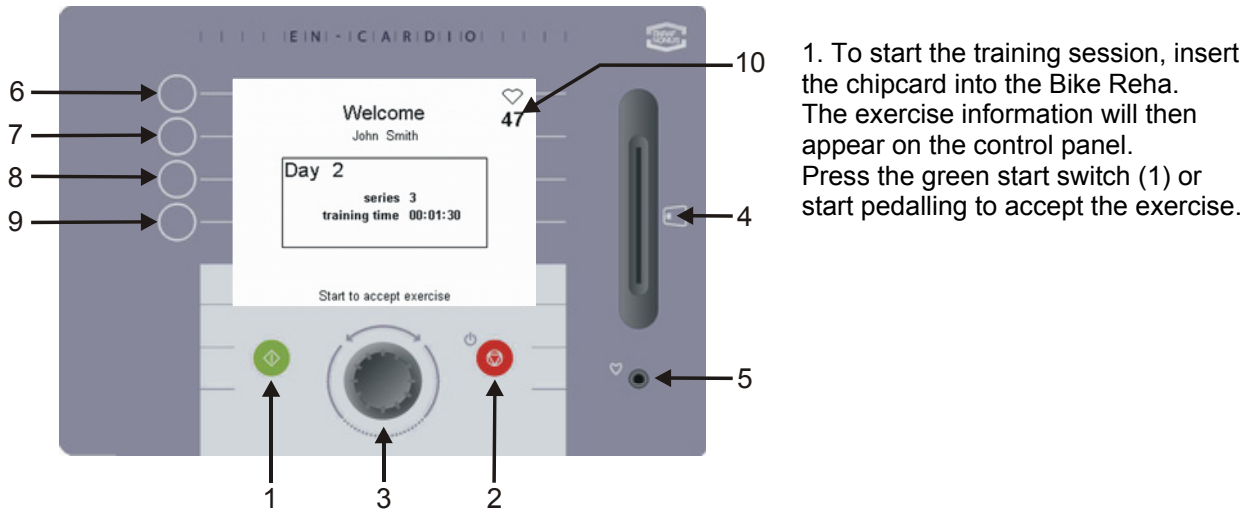
Fig. 16 Chipcard-controlled training



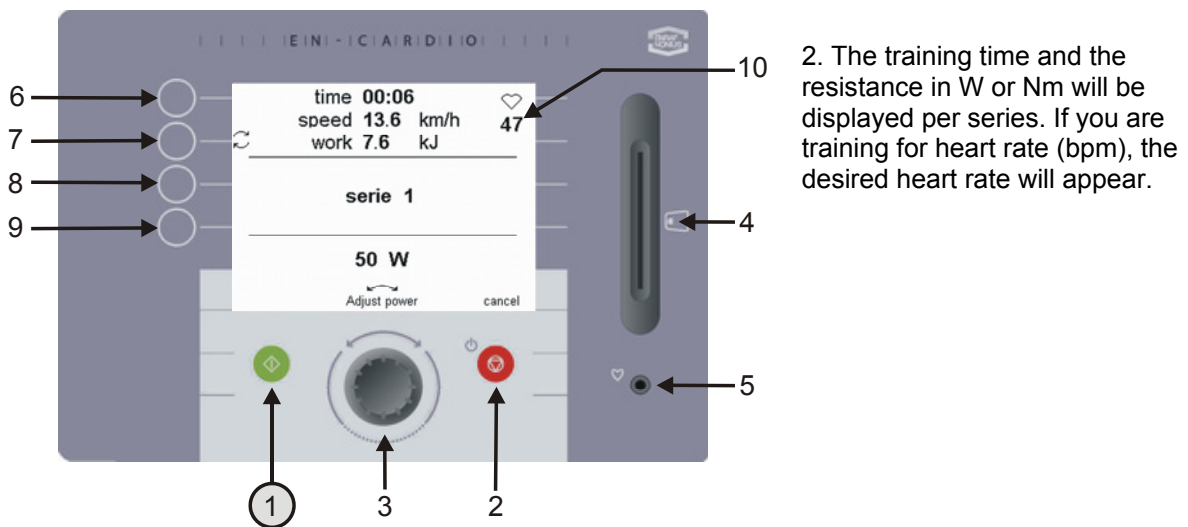
Fig. 17 Display of an EN-Track training session

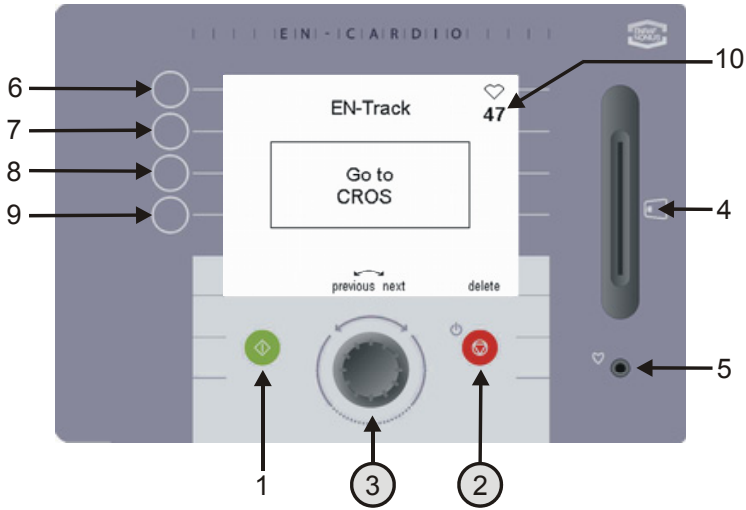3. At the end of each exercise the screen will show which other exercise you should proceed to next. (Fig. 18)  
 4. If you want to perform another exercise, you can turn the rotary button (3) to see which other exercises still have to be performed. This is only possible, however, if the exercise sequence is 'freely' adjustable in the EN-Track software. By pressing the delete button (2) an exercise that you do not wish to perform will be deleted.

Fig. 18 Remaining exercises

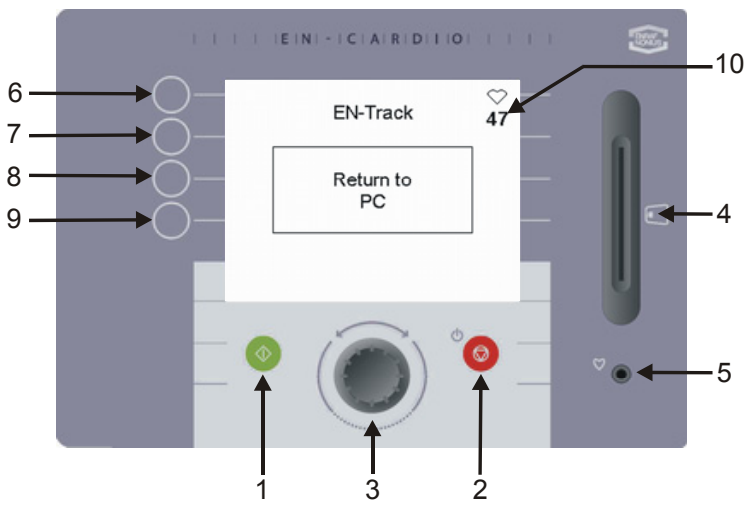5. When all the exercises have been performed, EN-Track will refer you to the PC. (Fig. 19)

Fig. 19 Returning to the PC

---

## 13 Product liability

---

A law on Product Liability has become effective in many countries. This Product Liability law implies, amongst other things, that once a period of 10 years has elapsed after a product has been brought into circulation, the manufacturer can no longer be held responsible for possible shortcomings of the product.

To the maximum extent permitted by applicable law, in no event will Enraf-Nonius or its suppliers or resellers be liable for any indirect, special, incidental or consequential damages arising from the use of or inability to use the product, including, without limitation, damages for loss of goodwill, work and productivity, computer failure or malfunction, or any and all other commercial damages or losses, even if advised of the possibility thereof, and regardless of the legal or equitable theory (contract, tort or otherwise) upon which the claim is based. In any case, Enraf-Nonius's entire liability under any provision of this agreement shall not exceed in the aggregate the sum of the fees paid for this product and fees for support of the product received by Enraf-Nonius under a separate support agreement (if any), with the exception of death or personal injury caused by the negligence of Enraf-Nonius to the extent applicable law prohibits the limitation of damages in such cases.

Enraf-Nonius cannot be held liable for any consequence resulting from incorrect information provided by its personnel, or errors incorporated in this manual and / or other accompanying documentation (including commercial documentation)

The opposing party (product's user or its representative) shall disclaim Enraf-Nonius from all claims arising from third parties, whatever nature or whatever relationship to the opposing party.

---

## 14 Maintenance

---

The Bike Reha is largely maintenance-free. The only point you really need to pay attention to is that wear residues from the flat belt need to be periodically removed from the large driving disc on the pedal shaft. The surface of the driving disc can be easily reached from underneath the machine without disassembly. After cleaning, the disc should be lightly rubbed with just one drop of oil.

To clean the membrane keyboard and the plastic housing, use only a soft, slightly moistened cloth. Do not use scourers or detergents for this purpose. These can damage the surface or even completely destroy it.

The warranty lapses if internal maintenance is performed by anyone who is not authorised by Enraf-Nonius.

- Always disconnect the Bike Reha from the mains before performing any maintenance work!
- If the power cord gets damaged, de-energise the device immediately using the switch (9). Next, remove the power cord from the socket to prevent injury.
- If general faults, fluctuations in resistance or faults in the control panel occur then stop the training immediately and switch off the device.
- Make sure that the various adjustable parts (saddle, handlebar) are firm and tight.
- The used lever-clamp systems (for example the seat height adjustment lever) should be lubricated with a drop of oil at regular intervals. This will ensure proper functioning
- The saddle pin of the Bike Reha should be rinsed at regular intervals and then be coated with some Silicon spray. This will result in a smooth saddle height adjustment mechanism.

**Please find out what your local rules and regulations are for the disposal of equipment and accessories.**

---

## 15 Conclusion

---

We hope that you have many years of enjoyable use of the Bike Reha ahead of you. We would like to draw your attention to the other high-quality products in our assortment. If you have any questions or would like to receive our catalogue, then please do not hesitate to contact our representative.

## 16 Technical details

Mains voltage	: 230 V
Power supply	: via power cord (230 V EUR)
Frequency	: 50 Hz
Permitted voltage variation	: +/- 10%
Power consumption	: approx. 40VA
Braking principle	: process-controlled eddy-current brake
Load	: 20-400 W (resolution: 5 W increments) rpm-independently from 35 – 110 rpm : 6 – 60 Nm (resolution: 0.5 Nm increments) rpm-independently from 35 – 110 rpm
Accuracy of rpm	: rpm, revs +/- 2 min <sup>-1</sup> : power in Watts, +/- 5% or 3 W. (depending on which value is the larger) See also the range in Fig. 20.
Heart rate measurement accuracy	: heart rate measurement with Polar®, +/-2% or 1 beat per minutes (depending on which value is the larger) This definition assumes constant conditions. Heart rate measurement with ear clip +/- 5% This definition assumes constant conditions.
Min. adjustable power	: 20 W
Medical class	: class IIa
Accuracy class	: class SA (EN 957)
Electrical safety	: Class II type B (3446665 & 3446676) or BF (3446670 & 3446675) (according to IEC 60601-1)
Dimensions	: 118 cm x 115 cm x 55 cm (l x h x w)
Weight	: approx. 55 kg
Saddle height (adjustable)	: 50 - 110 cm (EN975-1 and -5)
Steering column height (adjustable)	: 75 - 115 cm
Crank length	: 33.5 ± 1.5 cm
Max. patient weight	: 150 kg / 330 lbs
Display	: LCD with backlighting
Languages	: English, Dutch, German
Heart rate monitoring	: Polar® chest strap and receiver and hand/pulse measurement (options) Ear sensor prepared.
Exercise modes	: constant torque, constant power, constant heart rate
Parameters displayed during training	: time, heart rate, speed, distance, work, power and resistance

### Operating conditions:

Temperature	: 18 - 40 °C
Humidity	: 10 - 85% at 25 - 40 °C, non-condensing
Air pressure	: 560 - 1500 hPa

### Storage and transport conditions:

Temperature	: 14 - 40 °C
Humidity	: 30 - 95% at 25 - 40 °C, non-condensing
Air pressure	: 560 - 1500 hPa

### Program capacity

Fixed programs	: 12 fixed exercise profiles
----------------	------------------------------

### Installation of the Bike Reha with chipcard function on the EN-Track software.

You must have at least version 6.5 of the EN-Track software to be able to operate the machine in a chip-controlled manner.

The patient shall receive instructions to stop training when prescribed environmental conditions are not met.

Technical changes reserved.

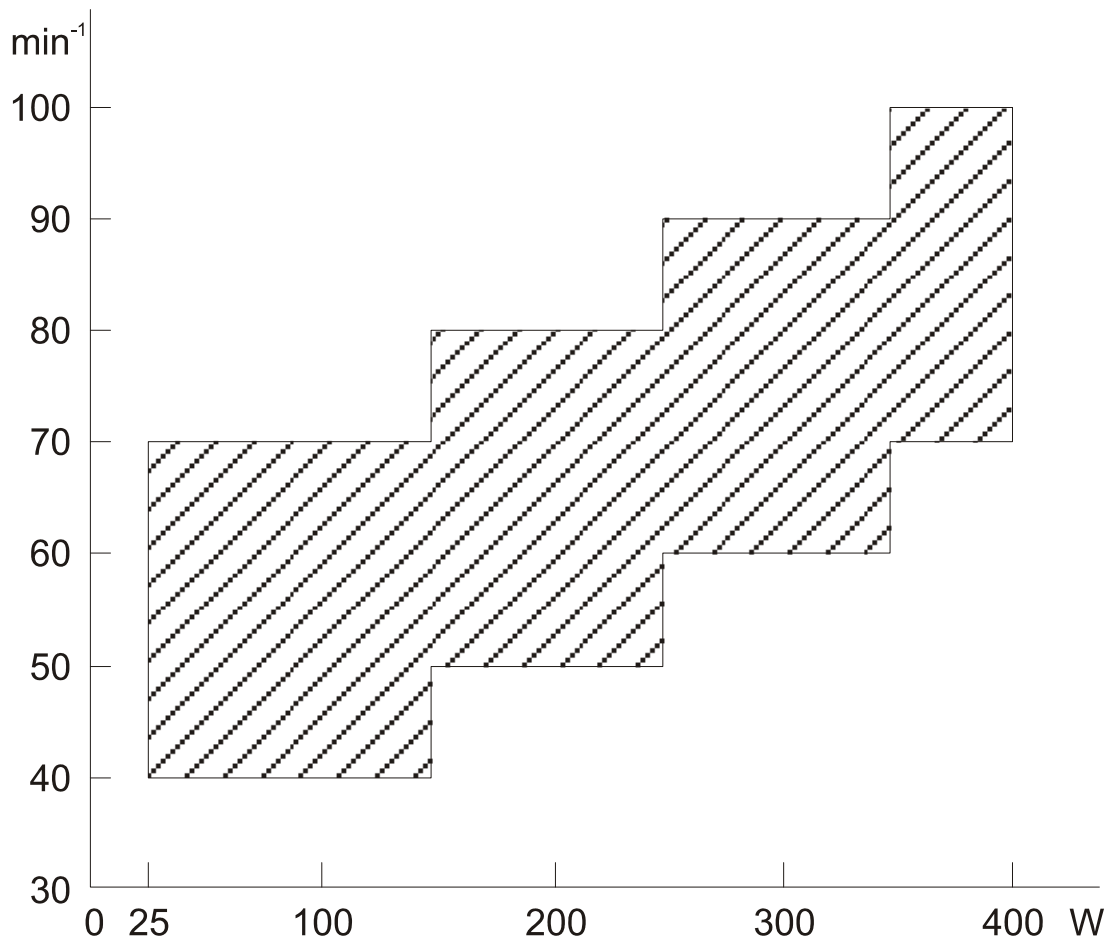
Fig. 20 Accuracy of rpm

---

## 17 Ordering details

---

- 3446665 Bike Reha
- 3446670 Bike Reha incl. hand/pulse measurement
- 3446675 Bike Reha Track incl. hand/pulse measurement and adjustable cranks
- 3446676 Bike Reha Track incl. adjustable cranks

The following accessories for the Bike Reha are available:

- 3496161 Bookstand
- 3496839 Polar® chest strap for heart rate monitoring
- 3496840 Elastic band for Polar® chest strap for heart rate monitoring

As options the following are available with a new Bike Reha:

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