
# Crosswalker

Bedieningshandleiding  
Operating Instructions  
Gebrauchsanweisung  
Mode d'emploi  
Instrucciones de manejo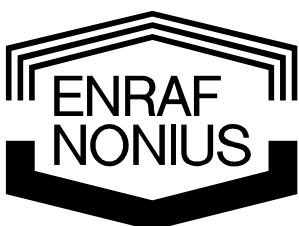

Manufacturer:

Milon Industries GmbH  
An der Laugna 2  
86494 Emersacker  
Germany

Copyright: & Exclusive Sales and Service

Enraf-Nonius B.V.  
P.O. Box 12080  
3004 GB Rotterdam  
The Netherlands  
Tel.: +31-(0)10 – 20 30 600  
Fax: +31-(0)10 – 20 30 692  
info@enraf-nonius.nl  
[www.enraf-nonius.com](http://www.enraf-nonius.com)

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## TABLE OF CONTENTS

<b>1</b>	<b>Foreword</b> .....	<b>27</b>
<b>2</b>	<b>Set-up</b> .....	<b>27</b>
	2.1 Placement.....	27
	2.2 Electrical connection.....	27
	2.3 Initial use.....	27
	2.4 Environmental information.....	27
<b>3</b>	<b>Safety instructions</b> .....	<b>28</b>
	3.1 Safety aspects.....	28
	3.2 Legal aspects.....	28
	3.2.1 Warnings.....	28
	3.2.2 Caution:.....	28
	3.2.3 Note:.....	29
<b>4</b>	<b>General</b> .....	<b>30</b>
	4.1 Parts list.....	30
	4.2 Control panel.....	31
	4.3 LCD screen.....	31
	4.4 Menus.....	31
	4.5 Heart rate monitoring.....	31
	4.5.1 Ear sensor.....	32
	4.5.2 Chest strap.....	32
	4.5.3 Hand/pulse measurement.....	33
	4.6 Adjusting the Crosswalker.....	33
<b>5</b>	<b>Contra-indications</b> .....	<b>33</b>
	5.1 Absolute contra-indications:.....	33
	5.2 Relative contra-indications:.....	33
<b>6</b>	<b>Intended use</b> .....	<b>33</b>
<b>7</b>	<b>Operation in general</b> .....	<b>34</b>
	7.1 Manual operation.....	34
	7.1.1 Constant power (W).....	34
	7.1.2 Constant torque (Nm).....	34
	7.1.3 Constant heart rate (bpm).....	34
<b>8</b>	<b>Quick-start</b> .....	<b>35</b>
	8.1 Quick-start in manual operation.....	35
<b>9</b>	<b>Manual operation</b> .....	<b>36</b>
	9.1 Configuring a profile.....	36
<b>10</b>	<b>Results</b> .....	<b>38</b>
<b>11</b>	<b>Changing the preferences</b> .....	<b>39</b>
<b>12</b>	<b>Using the Crosswalker with EN-Track software</b> .....	<b>41</b>
	12.1 General.....	41
	12.2 Possibilities with EN-Track.....	41
<b>13</b>	<b>Product liability</b> .....	<b>43</b>
<b>14</b>	<b>Maintenance</b> .....	<b>43</b>
<b>15</b>	<b>Conclusion</b> .....	<b>43</b>
<b>16</b>	<b>Technical details</b> .....	<b>44</b>
<b>17</b>	<b>Ordering details</b> .....	<b>45</b>

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## 1 Foreword

---

Thank you for purchasing the Crosswalker. This ergometer will bring great flexibility to your cardiovascular training routines.

The packaging contains 1) the Crosswalker, 2) these Operating Instructions.

In general, six of our products can be used for cardiovascular training. In addition to the Crosswalker (fig. 1, page 29), these include the Step, the Stair, the Bike Reha, the Cruiser and the EN-Mill treadmill.

When designing this ergometer a great deal of work went into ensuring that it was reliable, safe, user-friendly and durable. To ensure that you use your Crosswalker properly and safely it is important that you read through the following instructions *before* you start using it.

---

## 2 Set-up

---

### 2.1 Placement

If the floor is uneven then the height of the two rubber feet on the crossbars can be adjusted to compensate. These feet can easily be rotated and fixed into position using suitable tools.

### 2.2 Electrical connection

To supply power to the electronics a protective earthing 230V socket is required.

### 2.3 Initial use

After placing the Crosswalker in position you must wait for several hours before you can use it for the first time! This is necessary because of possible condensation caused by large differences between internal and external temperatures.

### 2.4 Environmental information

During the use of the Crosswalker no emissions appear that are outside the permissible range and that will cause any environmental burden.

Your Crosswalker contains no materials which are noxious for the environment. As thus, at the end of the duration of life, no special concern is needed. However, specialised companies can take apart the Crosswalker and sort out the materials for recycling. By doing so you contribute to a better environment.

**Please ensure that you are well informed of the local rules and regulations regarding to the removal of equipment and accessories.**

## 3 Safety instructions

### 3.1 Safety aspects

Safe execution of the procedures in this manual requires technical experience and a general knowledge of safety precautions as well as a general knowledge of the human locomotor system.

Warnings, Cautions and Notes have been used throughout the manual to bring special matters to the reader's immediate attention.

- A **Warning** concerns danger to the safety of the user or technician;
- A **Caution** concerns an act that might damage the equipment;
- A **Note** highlights a statement that deserves more emphasis than the general text, but which does not merit a **Warning** or a **Caution**.

It is important to follow the correct sequence of Crosswalkers in a procedure to ensure safe handling of the equipment and the prevention of damage. Never, therefore, change the sequence of a procedure.

### 3.2 Legal aspects

This manual and the information it contains are the property of Enraf-Nonius B.V. (The Netherlands.) Enraf-Nonius B.V. disclaims any responsibility for personal injury and/or damage to equipment caused by:

- Negligence or disregard of a Warning or a Caution.
- Deviation from any of the described procedures.
- Performance of activities that are not described in this manual.
- Ignorance of the general safety precautions for handling tools and the use of electricity.
- Usage of the device in a manner other than that specified in this manual.

#### 3.2.1 Warnings

- This cardiovascular training equipment is intended for use under (para)medical supervision.
- The operator shall take notice of the physical and mental condition of the patient and, to his judgement, decide whether exercising is admissible.
- Stop the training **immediately** if your patient feels unwell or dizzy, or if he/she experiences symptoms of cardiac pathology!
- Check that you patient is complying with the pre-set maximum heart rate limit, and stop the training immediately if this limit is exceeded.
- When using the Polar® chest strap, a sufficient distance (at least 1.5 metres) must be allowed between users. If not, there is a risk that the pulse measurement will be distorted.
- In case of a dermatological condition, injured skin shall never contact the keyboard, heart-rate sensor, ear sensor, seat, handlebar or pedals.
- In case of patiënts with cardiac pathology are going to train on your Crosswalker than the integrated heartrate sensor should not be used. Heartrate measurement by means of an ECG system is more appropriate.

#### 3.2.2 Caution:

- A check shall be carried out on the packaging and on the device and for deterioration, before putting the Crosswalker into use.
- The Crosswalker must be connected to a 230V/50Hz outlet. If the wrong voltage is used then the Crosswalker can be damaged or may function incorrectly.
- The Crosswalker shall not be installed in a location near a heat source. Exposure to direct sunlight shall be avoided.
- The device shall not be used in close proximity (i.e. less than 5 metres) of short-wave and microwave equipment.
- Never insert your hand under the cover! Through the open underside you can come into contact with the drive mechanism, the belt or the brake.
- When **any** maintenance or service work is being performed and whenever the side-cover of the housing is removed the Crosswalker **must** be **disconnected from the mains!**
- An uneven surface can adversely affect the generally very high level of **stability** of the Crosswalker. Correct adjustment of the rubber feet can **guarantee** the Crosswalker's stability.

- The Crosswalker is a specialised device for blood circulation and ergometer training! Inexpert use can damage the Crosswalker.
- The patient shall receive instructions for correct operation of the Crosswalker.
- Using the Crosswalker in a manner other than that described in these Instructions can lead to hazardous situations. You are advised to read through these Instructions carefully before starting to use the Crosswalker.
- If the power cord gets damaged, remove the power cord from the socket to prevent injury.
- If general faults, fluctuations in resistance or faults in the control panel occur then stop the training immediately and switch off the device.
- Keep unsupervised children away from the training equipment at all times.
- The user must assume a correct biomechanical position.
- Training incorrectly can lead to injury. Supervision by an expert (para)medic is therefore essential.
- The Crosswalker shall not be used for diagnostic purposes
- The apparatus may not be used in so-called “wet rooms” (hydrotherapy rooms)
- The apparatus should only be used by patient within the permissible weight range as indicated in the technical specifications
- The heartrate monitoring system with earclip or hand/wrist measurement is only intended for heartrate measurement. For heartrate targeted training only the **Polar®** system with chestbelt is allowed
- A daily pre-use check shall be carried out on:
  - the heart rate measuring sensors
  - the moving parts
- In case of a dermatological condition, injured skin shall never contact the keyboard, heart-rate sensor, ear sensor, hand sensor, saddle, handlebar or pedals.

**Note: all parts that come in contact with the patients should be cleaned and disinfected (EEC No. 603-000-00-5).**

### 3.2.3 Note:

- If any faults should arise in your Crosswalker, please contact your equipment supplier or Enraf-Nonius B.V. (The Netherlands.) Any repairs and installations should only be performed by a service technician who has been authorised by Enraf-Nonius.
- Repairs performed by service technicians who are not authorised by Enraf-Nonius and/or the use of non-original (replacement) parts and/or auxiliary equipment invalidates the warranty and can be hazardous.
- An annual service check is necessary. Technical maintenance shall only be done by personnel, authorized by Enraf-Nonius. A maintenance checklist is available in the service manual.

## 4 General

The Crosswalker can be used for various purposes, ranging from fitness to rehabilitation.

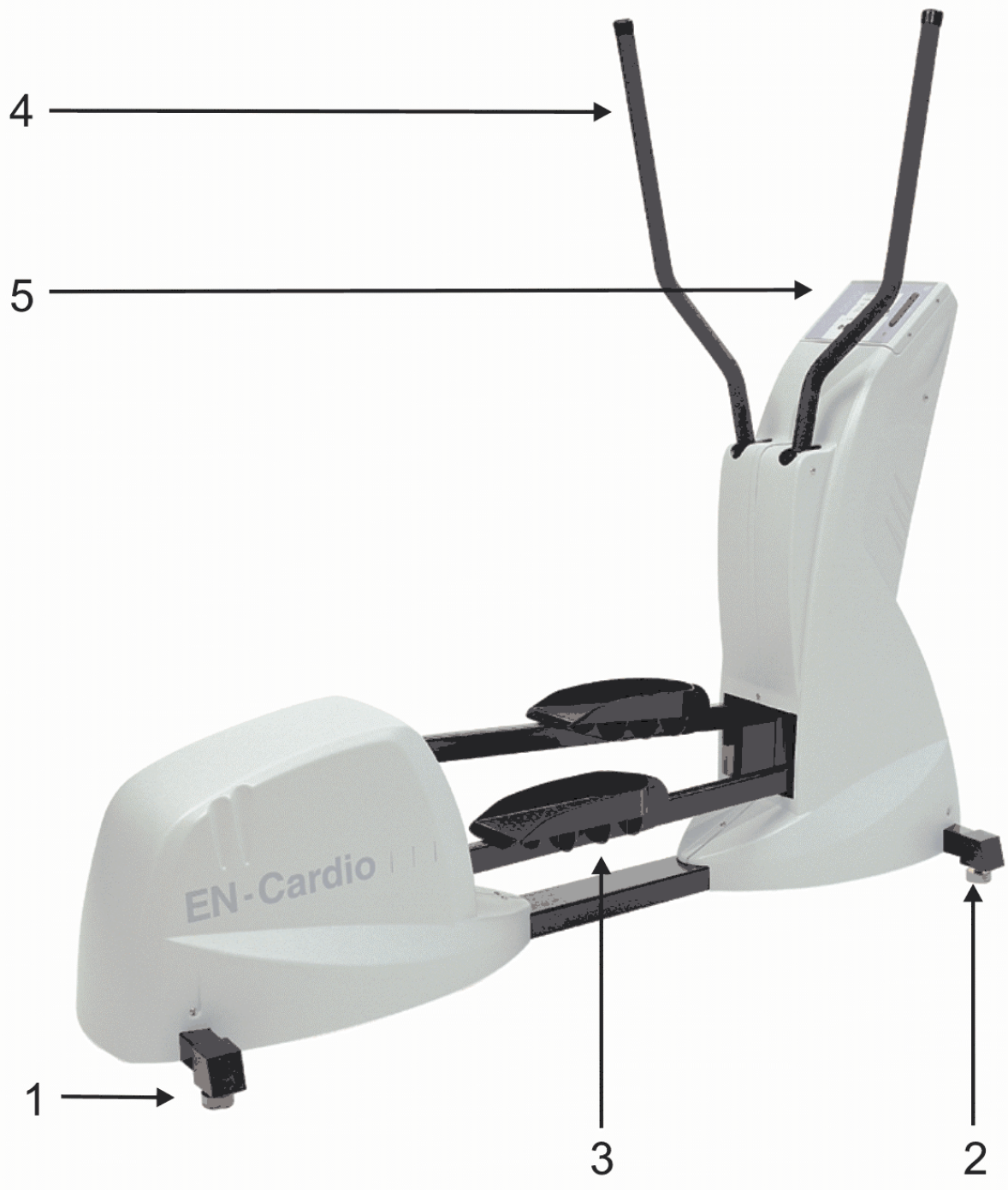
Fig. 1. Crosswalker

### 4.1 Parts list

1. Crosswalker rear support
2. Crosswalker front support
3. Pedals
4. Handlebars
5. Control panel

## 4.2 Control panel

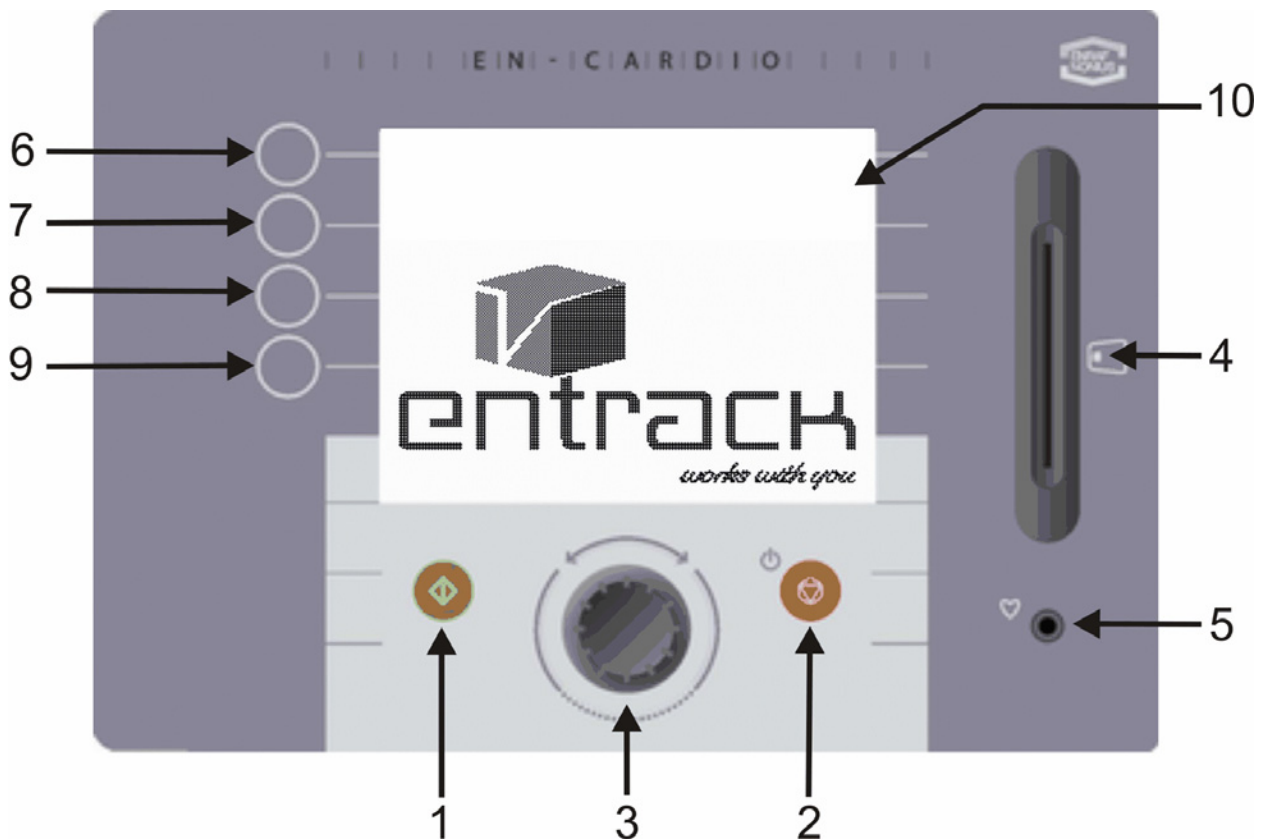
Fig. 2. Control panel

The control panel of the Crosswalker consists of the following: (Fig. 2)

1. **Start** switch to turn the Crosswalker on
2. **Stop** switch to turn the Crosswalker off
3. Rotary button for various adjustments
4. Chipcard reader
5. Ear clip for heart rate receiver
6. Selection switch 1
7. Selection switch 2
8. Selection switch 3
9. Selection switch 4
10. LCD screen

## 4.3 LCD screen

The control panel (Fig. 2) contains an LCD screen. During the preparations for a training session the control panel displays the menu. During the session itself it shows the various parameters and a graphical representation of the session.

## 4.4 Menus

The Crosswalker is menu-driven. In the menu you can select various training modes. The Crosswalker can be operated with the switches and the rotary button on the control panel. You will find a detailed explanation of the various menus in chapters 5 to 10 of these Instructions.

## 4.5 Heart rate monitoring

There are 3 ways of monitoring the heart rate:

1. ear sensor
2. chest strap (option)
3. hand/pulse measurement (option)

#### 4.5.1 Ear sensor

Various ear sensors that exist on the market can be connected to the Crosswalker.

However, the Enraf-Nonius program does not include ear sensors since we can not guarantee proper functioning of such sensors when measuring heart rate frequency.

For accurate heart rate measurement we strongly advise our users to make use of the Polar® chest belt (optional accessory).

Use the ear sensor to measure your heart rate as follows:

- Attach the ear sensor to your ear lobe.
- Attach the collar clip to your collar or shirt. This stops the sensor from moving or being disturbed during use. The wire of the ear sensor must be sufficiently long.
- The sensor can be attached to the ear lobe or (even better) can be inserted into the shell of the ear.

**After usage, clean the sensor according to the instructions of the manufacturer. For most sensors a mild soap solution is sufficient. Do not use detergents!**

#### 4.5.2 Chest strap

While training on the Crosswalker, a Polar® heart rate monitoring system can also be used. The Polar® heart rate monitoring system consists of a lightweight transmitter strap worn round the chest and a receiver which is inserted into the control panel of the Crosswalker. The transmitter uses two electrodes to record each heartbeat and then sends the signals to the receiver. During a training session the heart rate appears in the upper right-hand corner of the screen.

Use the chest strap to measure your heart rate as follows:

1. Attach the transmitter to the elastic band

2. The chest strap must be comfortable and must fit the user's chest snugly

3. Moisten the grooved rear surface of the transmitter, where the electrodes are located

4. Check that the part with the transmitter is positioned correctly, i.e. on the right-hand side with the Polar logo in the middle

**Disconnect the ear sensor from the control panel. If you do not do so then there will be no telemetric monitoring via the chest strap.**

The chest strap works best if the transmitter is worn against the bare skin. If you prefer to wear the transmitter over a shirt then you must also moisten the part of transmitter the where the grooved surfaces rest against the shirt.

Maintenance of the chest strap:

- Clean the transmitter after use with a mild soap solution
- Rinse with clean water
- Dry the transmitter thoroughly, otherwise the internal batteries will run down

### 4.5.3 Hand/pulse measurement

During training on the Crosswalker, a Polar® hand/pulse monitoring system can also be used (optional). First disconnect the ear sensor from the control panel. Grasp the two hand/pulse sensors that are attached to the steering column firmly in both hands. When a pulse value is measured this will appear in the top right-hand corner of the control panel. It can take some time (about 20 seconds) before the measured pulse value appears on the screen. Throughout the training session keep your hands on the two hand/pulse sensors.

### 4.6 Adjusting the Crosswalker

The Crosswalker is optimally suited to every patient on delivery. No further adjustments are required.

---

## 5 *Contra-indications*

---

### 5.1 Absolute contra-indications:

- Myocardial problems in an unstable phase
- HNP in the (sub-)acute stage (with radiation and neurological abnormalities)
- Severe osteoporosis
- Spontaneous fractures
- Severe mechanical instability
- Early postoperative joint modifications

### 5.2 Relative contra-indications:

- Myocardial problems in a stable phase
- HNP in the subacute stage (without radiation and neurological abnormalities)
- Mild osteoporosis
- Postoperative joint modifications
- Tumours
- Inflammations in the area to be activated
- Undesirable articular load in the area to be activated (e.g. in M. Osgood-Schlatter, bursites, etc.)
- Undesirable ligamentous load in the area to be activated (e.g. overstretching with swelling)
- Undesirable soft tissue loading in the area to be activated (e.g. in very severe atrophy)
- Cardiovascular limitations
- Pulmonary limitations
- Psychogenic limitations
- Pain of such a nature that movement does not promote recovery

---

## 6 *Intended use*

---

The Crosswalker is an ergometer for medical training purposes.

As thus the Crosswalker can be used for:

- Prevention: regular use should prevent cardio-respiratory and locomotor impairments.
- Rehabilitation: muscular dysbalance can be adjusted / equalized by means of regular use
- Diagnostic aid: by means of assessment of the changes in heartrate during exercise and evaluation can be made with respect to the power producing capability of the patient.

## 7 Operation in general

This chapter describes how the Crosswalker is used **without** the chipcard function. All the machines in the EN-Cardio group have a uniform operation. For a description of how EN-Track is used **with** the chipcard function, please consult chapter 11.

The Crosswalker must be operated under the supervision of the physiotherapist or other suitably qualified person. Only the therapist should attempt to interpret the results of the training session.

### 7.1 Manual operation

The Crosswalker can be operated manually. All you need to do is decide which form of exercise you want to execute.

The options are:

- Constant power
- Constant torque
- Constant heart rate (pulse)

#### 7.1.1 Constant power (W)

This position ensures that the power remains constant irrespective of the Crosswalker speed and heart rate: the power (20-400W) can be increased or reduced using the rotary button (even during the training) in 5W increments.

#### 7.1.2 Constant torque (Nm)

The resistance will be maintained at a constant level so that a constant torque (constant resistance) is created. The value of a constant torque (5-60 Nm) can be increased or reduced using the rotary button (even during the training) in 0.5 Nm increments.

#### 7.1.3 Constant heart rate (bpm)


The ergometer will bring the patient's heart rate to the pre-set value and then keep it there. This is done by adjusting the power so that the pre-set heart rate (according to the signal from the Polar® chest strap, ear clip or the hand/pulse measurement) is achieved and maintained. The measured value is shown at the extreme right of the first line of the exercise screen, under the heart indicator. The desired value can be increased or reduced in 1 bpm increments (between 40 - 210 bpm).

During the training (in which a warm-up is recommended so that the heart rate does not rise too quickly) the pre-set desired heart rate can be changed by using the rotary button to set the new value. After the rotary button has been operated the desired value will also appear at the bottom-centre of the LCD screen.

## 8 Quick-start

### 8.1 Quick-start in manual operation

With the quick-start function you can start training immediately. Proceed as follows: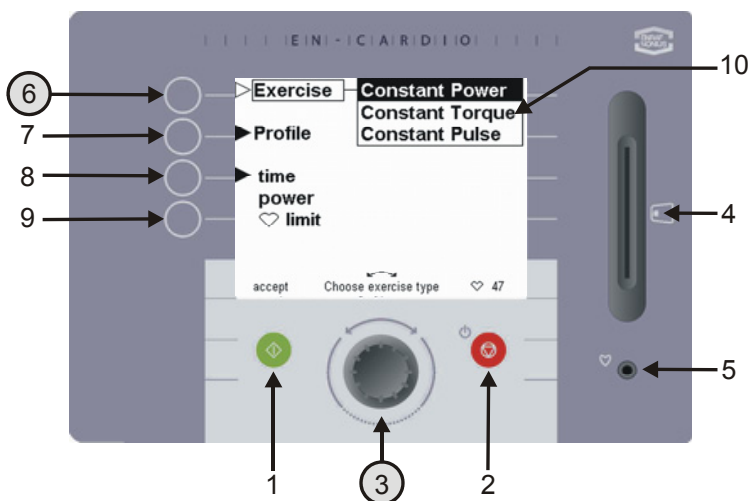


First use the rotary button (3) to select the desired exercise form under Exercise (6).

The options are:

- Constant power
- Constant torque
- Constant pulse

In the Exercise screen illustrated, 'constant power' has been selected.

Fig. 3. Exercise options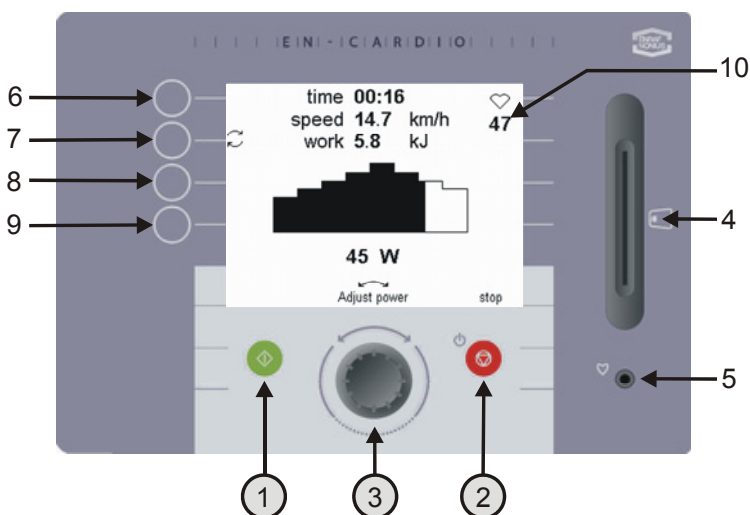


By pressing the green button (1) or starting to walk you start the training immediately in accordance with the values that have been pre-set under 'settings'. In the section on the 'preferences menu' (Chapter 9) you can read how these values can be changed.

During the training you can see the various parameters displayed on the exercise screen. The power can be adjusted with the rotary button (3). These parameters can also be adjusted when you training for torque or heart rate.

Press the red stop button (2) if you need to end a training session prematurely.

Fig. 4 Activating the quick-start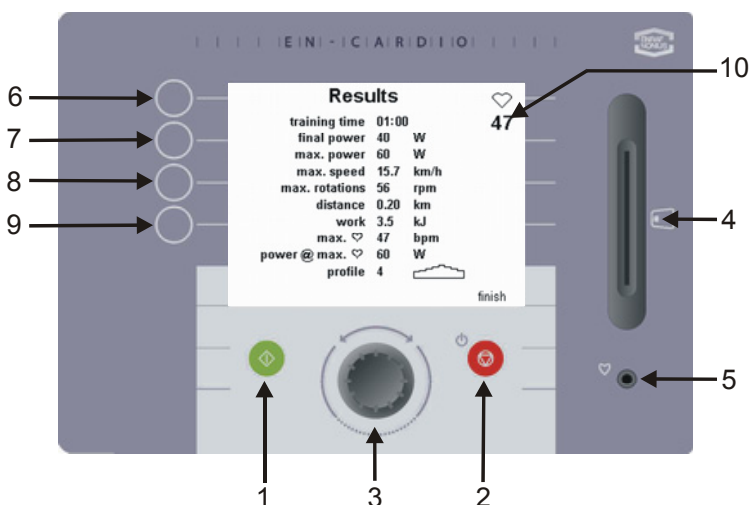

After the training session is over or has been prematurely terminated this screen will show the results of the session.

Fig. 5. Training results

## 9 Manual operation

### 9.1 Configuring a profile

On the Crosswalker you can pre-set constant power, constant torque or constant heart rate profiles. Below the procedure for configuring a constant power profile is discussed.

If you have selected a constant torque or a constant heart rate profile then you should follow the same Crosswalkers as described above.

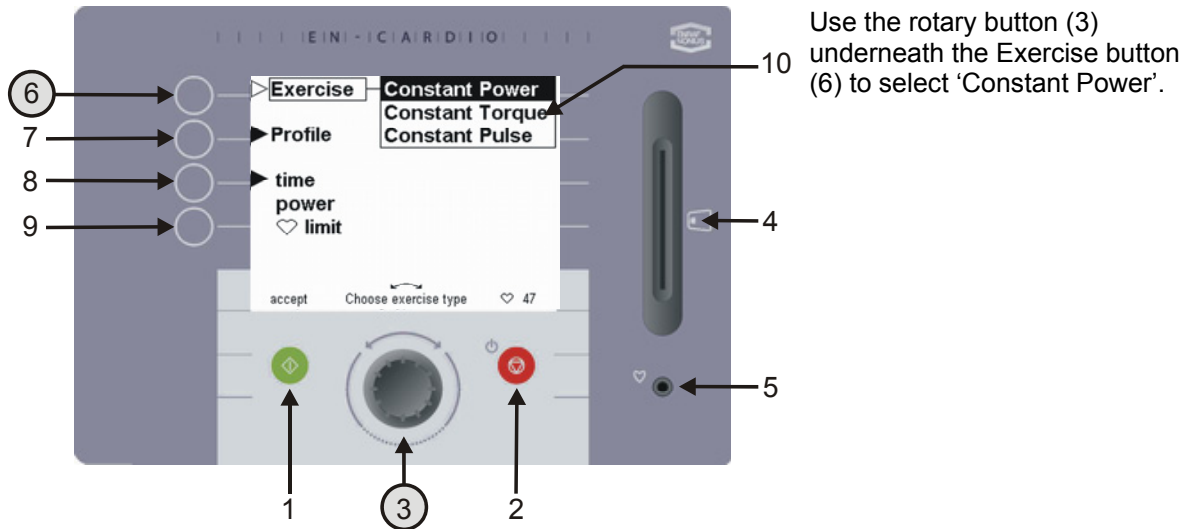
Fig. 6. Training options



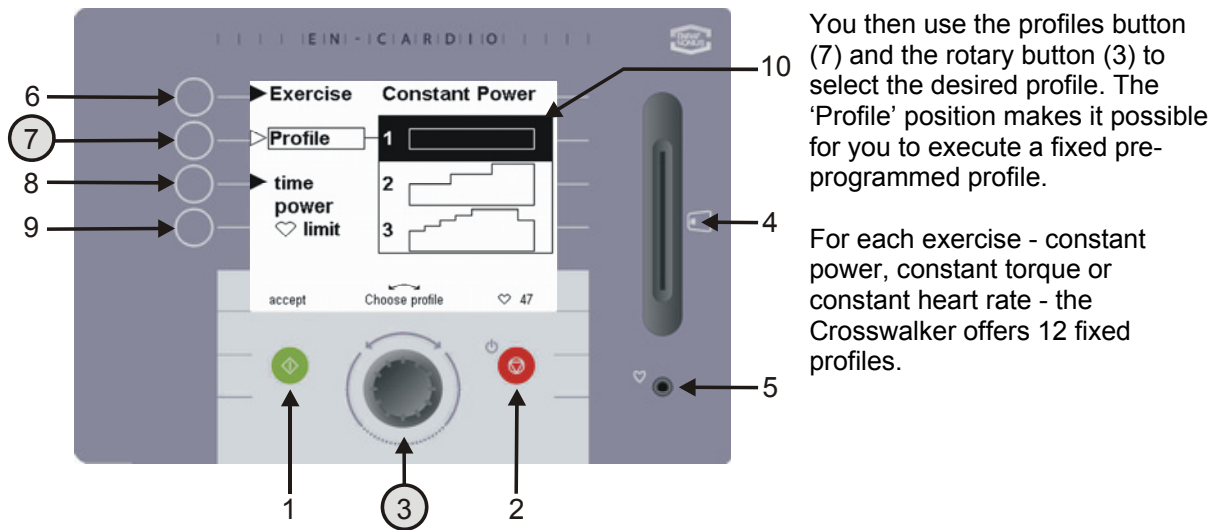
Fig. 7. Profile selection

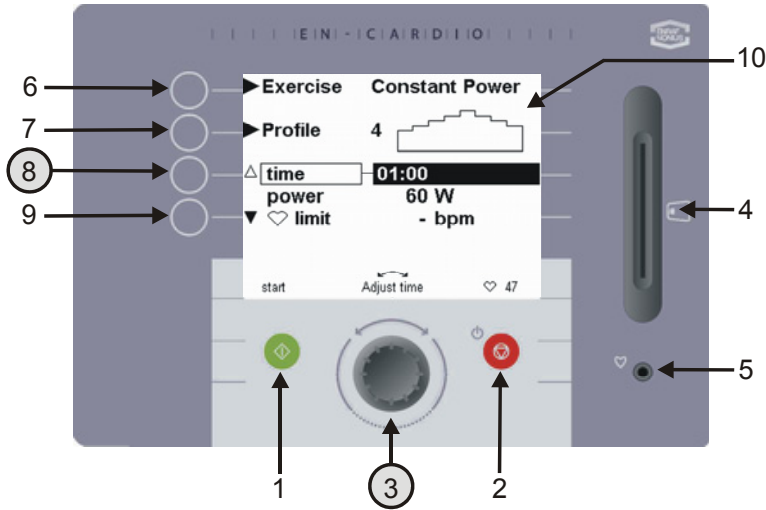
By pressing the Time button (8) you can set the duration of the exercise. The value for the time (between 00:30 seconds and 8:00 hours) can be changed with the rotary button (3). If a time of 0:00 is set then the training time will not expire but will instead run until a maximum time of 08:00 hours is reached.

Fig. 8 Entering the training time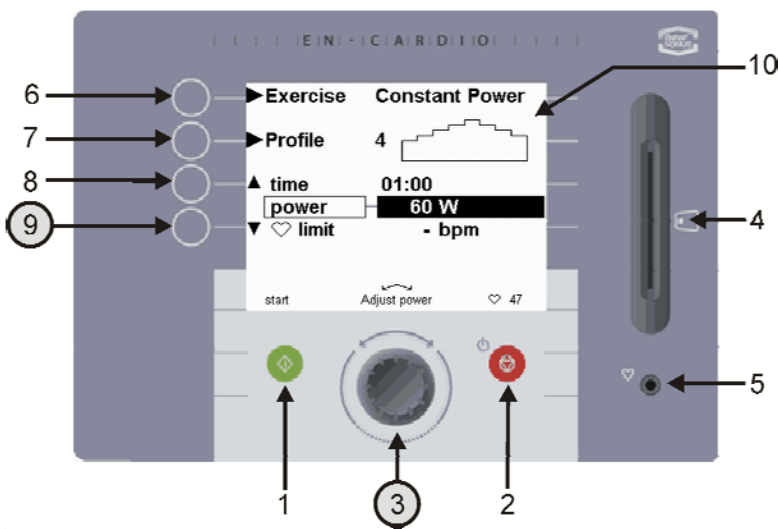

By pressing button 9 you can use the rotary button to set the maximum value of the selected profile. The starting value of this profile is half this value! If the pre-set value is 60 W, then the profile will start at 30 W. Allowance must be made for this when executing a constant heart rate profile. A gradual build-up of a heart rate profile is desirable, so make due allowance for this when selecting the various profiles!

Fig. 9 Entering the resistance (W)

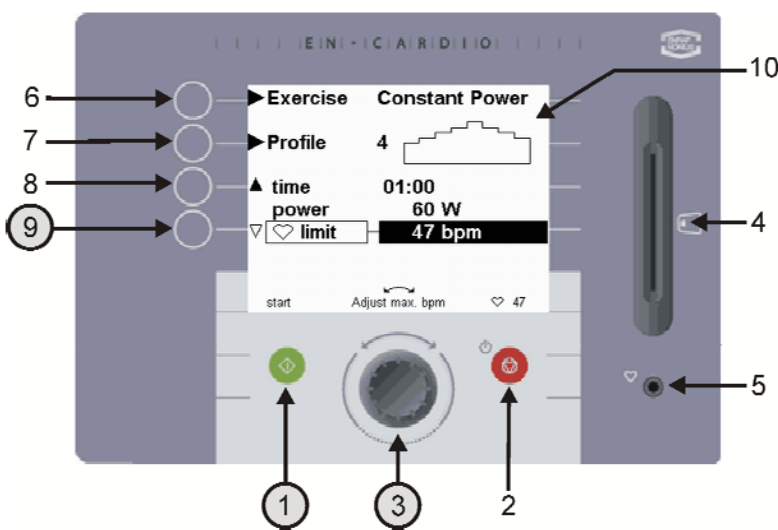
By pressing button 9 once again you will be able to use the rotary button (3) to pre-set a maximum heart rate limit (between 0 and 255 bpm) for the training session. This acts a maximum heart rate warning mechanism. In other words: if you are training for resistance or torque you can pre-set a warning for the maximum heart rate.

Fig. 10 Entering the maximum heart rate

There will be an acoustic and an optical warning if this maximum heart rate is exceeded. The resistance must be reduced by the user **himself** using the rotary button (3). The resistance is **not** adjusted by the Crosswalker, as is the case with the Constant Heart rate exercise form.

By pressing the green button (1) you start the training, and the exercise screen appears. (Fig. 11)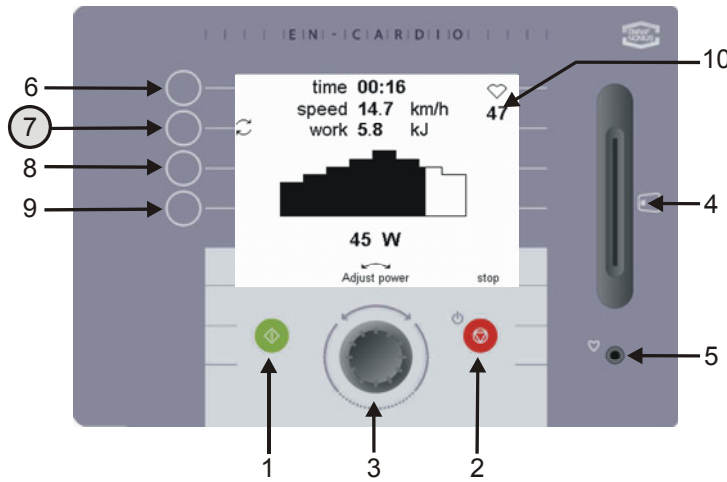

During the training session, seven different effort values can be displayed.

The following three values are continuously displayed:

**Time:** Training time remaining

**Speed:** Speed in [km/h] (this depends on the pre-set power in W)

**Heart Rate:** The heart rate is continuously displayed in Beats Per Minute [bpm] at the top-right of the LCD screen, under the ♥ symbol.

The following four values are shown alternately when button (7) is pressed.

The possible values are:

**Distance:** Distance covered in [km]

**Energy:** Effort produced in [kJ]


**Revs:** Pedal revolutions in revs per minute [rpm]

**Power:** Power, shown in [W]

Fig. 11 Changing the value displayed in the exercise screen

## 10 Results

When the training time has elapsed or if you have pressed the stop button (2) during the training, this overview screen will appear containing the results of the training just performed.

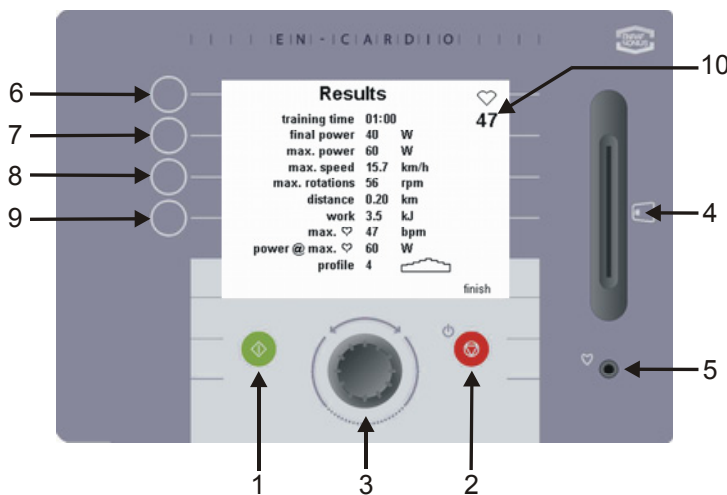The results displayed are:

- Training time .....min
- Final power.....Watt
- max. power .....Watt
- max. speed .....km/h
- max. rotations .....rpm
- distance.....km
- work.....kJ
- max. ♥.....bpm
- power @ max. ♥.....Watt (@ = at the moment of the max. heart beat)
- profile .....number

Fig. 12 Overview screen

## 11 Changing the preferences


You can use the 'changing preferences' function to change the manufacturer's settings. These changes can only be made by the therapist or coach. To access the preferences menu you need to perform a special action (this is to prevent changes being made by unauthorised persons).

If you want to change the manufacturer's (preference) settings you can do this in this menu.

An example of manufacturer's settings is the quick-start option (chapter 6)

The manufacturer's settings are as follows (see Fig. 14):

- 'Constant Power' exercise
- Profile 1
- Training time 5:00 min
- Etc.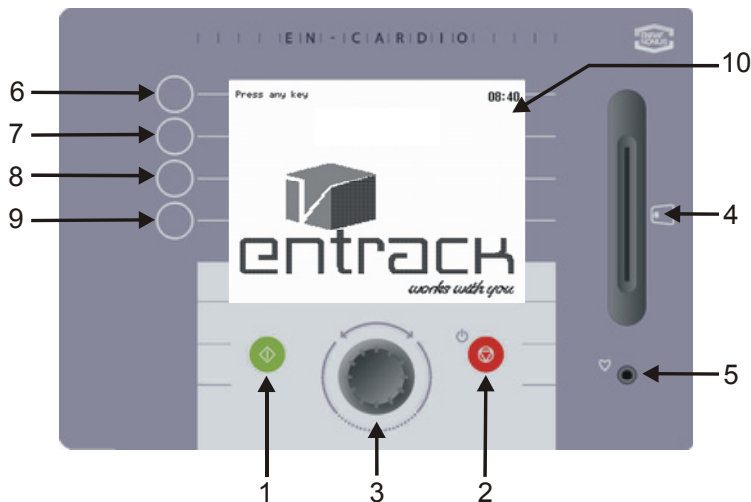

Access the preferences menu as follows:

1. Keep the stop button (2) on the Stand-by screen (fig. 13) depressed for 5 seconds.
2. When the device switches itself off, press buttons 6 and 9 simultaneously. Now the preferences menu appears. (Fig. 14)
3. In this menu you can change various settings using buttons 8 and 9. The pre-set values can be adjusted with the rotary button (3).

Fig. 13 Changing settings from the stand-by screen


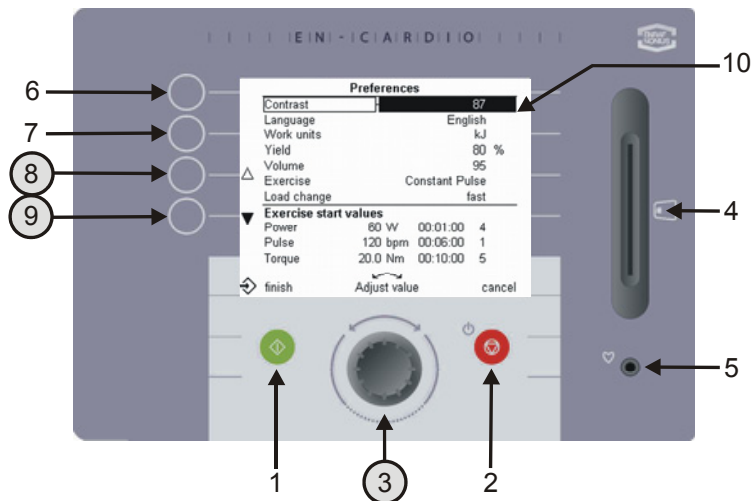
Fig. 14 Settings screen after entering the preference

The following settings can be changed with the rotary button (3):

- Contrast Set the display screen clarity
- Language Choose your own language
- Energy unit Choose between calories and joules
- Efficiency Energy burned by the client = pre-set power of the Crosswalker x efficiency %. The average value is 20%.
- Volume Set the desired volume
- Exercise Specify the form of exercise that begins with a quick-start
- Resistance change The transition in resistance can occur slowly, at average speed or quickly
- Year, month, day Enter the present date
- Time Enter the time
- Switch-off time Enter the time (5min - 60min), after which the machine will switch off
- Exercise start values Define the 'default settings' for the forms of exercise 'constant torque', constant power and constant heart rate for the following values:
  - resistance (W, Nm and bpm)
  - training time
  - profile number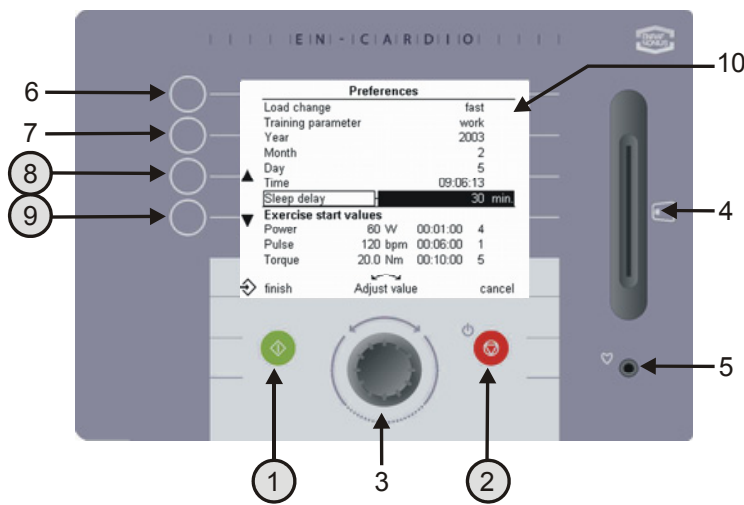

After making the changes you can exit the preferences menu by pressing the green exit button (1). The selected form of exercise, training time and profile number will then appear in the settings screen.

If you want to exit the screen **without** saving the changes, press the red break button (2). The settings screen appears.

Fig. 15 Preference settings

## 12 Using the Crosswalker with EN-Track software


### 12.1 General

The Crosswalker is an addition to the existing range of EN-Strength exercise machines. This part of the documentation describes the use of the Crosswalker with the EN-Track System, and gives some examples to make it easier to understand. To use the Crosswalker with the Track System you need to have a mastery of the EN-Track software, because controlling the Crosswalker is very similar to controlling the EN-Strength exercise machines. If you are unsure, consult the software help for EN-Track or the EN-Track software manual.

### 12.2 Possibilities with EN-Track


Using EN-Track enables you to execute more profiles on the Crosswalker. The number of profiles corresponds to the number of profiles you can create with the EN-Track system. The Crosswalker can be used in an exercise profile in a random sequence, just like the EN-Strength, EN-Free and EN-TreeP equipment.

To create a controlled training, you can select from a multiplicity of manually created training profiles. You can enter these on the chip-card via just a few actions. To create an exercise in the EN-Track software, select the tab EN-Cardio/Crosswalker exercises. For further explanation, please refer to the EN-Track software manual.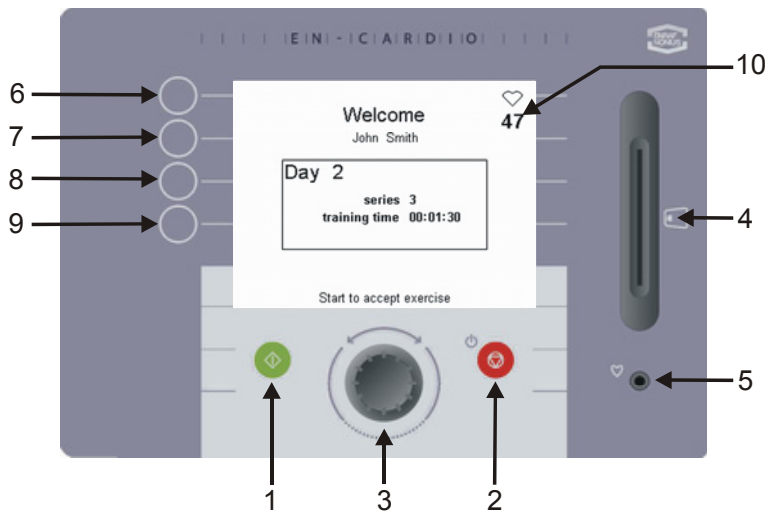


To start the training session, insert the chip-card into the Crosswalker. The exercise information will then appear on the control panel. Press the green start switch (1) or start walking to accept the exercise.

Fig. 16 Chipcard-controlled training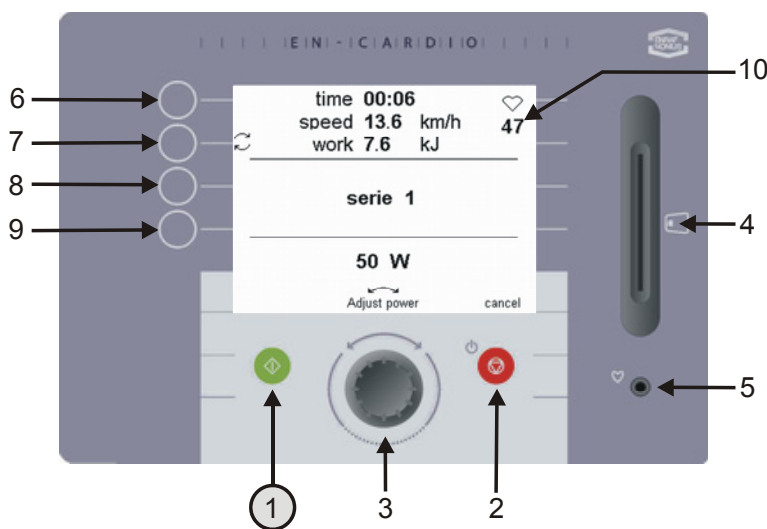

The training time and the resistance in W or Nm will be displayed per series. If you are training for heart rate (bpm), the desired heart rate will appear.


Fig. 17 Display of an EN-Track training session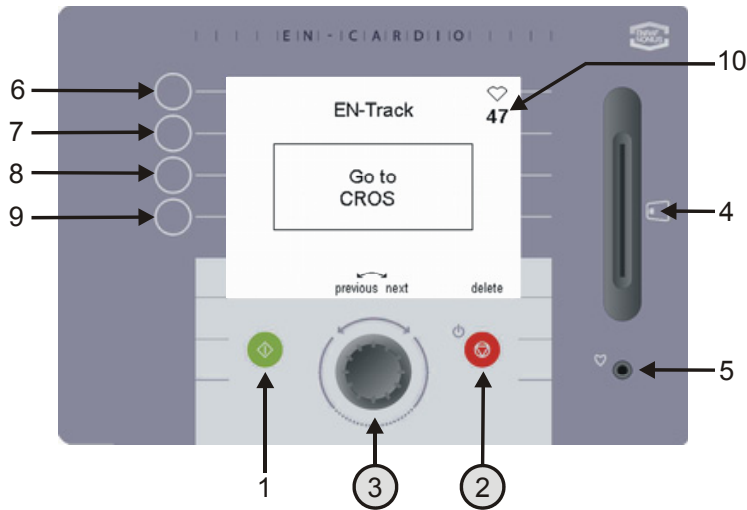

At the end of each exercise the screen will show which other exercise you should proceed to next.

If you want to perform another exercise, you can turn the rotary button (3) to see which other exercises still have to be performed. This is only possible, however, if the exercise sequence is 'freely' adjustable in the EN-Track software. By pressing the delete button (2) an exercise that you do not wish to perform will be deleted.

Fig. 18 Remaining exercises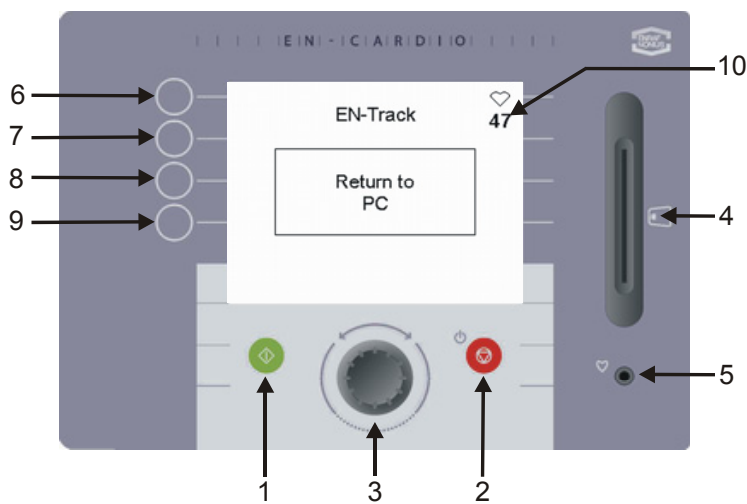

When all the exercises have been performed, EN-Track will refer you to the PC. (Fig. 19)

Fig. 19 Returning to the PC

---

## 13 Product liability

---

A law on Product Liability has become effective in many countries. This Product Liability law implies, amongst other things, that once a period of 10 years has elapsed after a product has been brought into circulation, the manufacturer can no longer be held responsible for possible shortcomings of the product.

To the maximum extent permitted by applicable law, in no event will Enraf-Nonius or its suppliers or resellers be liable for any indirect, special, incidental or consequential damages arising from the use of or inability to use the product, including, without limitation, damages for loss of goodwill, work and productivity, computer failure or malfunction, or any and all other commercial damages or losses, even if advised of the possibility thereof, and regardless of the legal or equitable theory (contract, tort or otherwise) upon which the claim is based. In any case, Enraf-Nonius's entire liability under any provision of this agreement shall not exceed in the aggregate the sum of the fees paid for this product and fees for support of the product received by Enraf-Nonius under a separate support agreement (if any), with the exception of death or personal injury caused by the negligence of Enraf-Nonius to the extent applicable law prohibits the limitation of damages in such cases.

Enraf-Nonius cannot be held liable for any consequence resulting from incorrect information provided by its personnel, or errors incorporated in this manual and / or other accompanying documentation (including commercial documentation)

The opposing party (product's user or its representative) shall disclaim Enraf-Nonius from all claims arising from third parties, whatever nature or whatever relationship to the opposing party.

---

## 14 Maintenance

---

The Crosswalker is largely maintenance-free.

To clean the membrane keyboard and the plastic housing, use only a soft, slightly moistened cloth. Do not use scourers or detergents for this purpose. These can damage the surface or even completely destroy it.

The warranty lapses if internal maintenance is performed by anyone who is not authorised by Enraf-Nonius.

- **Always** disconnect the Crosswalker from the mains before performing any maintenance work!
- If the power cord gets damaged than immediately remove the power cord from the socket to prevent injury.
- If general faults, fluctuations in resistance or faults in the control panel occur then stop the training immediately and switch off the device.

**Please find out what your local rules and regulations are for the disposal of equipment and accessories.**

---

## 15 Conclusion

---

We hope that you have many years of enjoyable use of the Crosswalker ahead of you. We would like to draw your attention to the other high-quality products in our assortment. If you have any questions or would like to receive our catalogue, then please do not hesitate to contact our representative.

## 16 Technical details

Mains voltage	: 230 V
Power supply	: via power cord (230 V EUR)
Frequency	: 50 Hz
Permitted voltage variation	: +/- 10%
Power consumption	: approx. 40VA
Braking principle	: process-controlled eddy-current brake
Load	: 20-400 W (resolution: 5 W increments) rpm-independently from 35 – 100 rpm : 6 – 60 Nm (resolution: 0.5 Nm increments) rpm-independently from 35 – 100 rpm
Accuracy of rpm	: rpm, revs +/- 2 min <sup>-1</sup> : power in Watts, +/- 10% or 5 W. (depending on which value is the larger) See also the range in Fig. 20.
Heart rate measurement accuracy	: heart rate measurement with Polar®, +/-2% or 1 beat per minutes (depending on which value is the larger) This definition assumes constant conditions. Heart rate measurement with ear clip +/- 5% This definition assumes constant conditions.
Min. adjustable power	: 20 W
Medical class	: class IIa
Accuracy class	: class SA (EN 957)
Electrical safety	: Class II type B (3446668) or BF (3446673) (according to IEC 60601-1)
Dimensions	: 190 cm x 177 cm x 55 cm (l x h x w)
Weight	: approx. 80 kg
Max. patient weight	: 150 kg / 330 lbs
Display	: LCD with backlighting
Languages	: English, Dutch, German
Heart rate monitoring	: Polar® chest strap and receiver and hand/pulse measurement (options) Ear sensor prepared.
Exercise modes	: constant torque, constant power, constant heart rate
Parameters displayed during training	: time, heart rate, speed, distance, work, power and resistance

### Operating conditions:

Temperature	: 18 - 40 °C
Humidity	: 10 - 85% at 25 - 40 °C, non-condensing
Air pressure	: 560 - 1500 hPa

### Storage and transport conditions:

Temperature	: 14 - 40 °C
Humidity	: 30 - 95% at 25 - 40 °C, non-condensing
Air pressure	: 560 - 1500 hPa

### Program capacity

Fixed programs	: 12 fixed exercise profiles
----------------	------------------------------

### Installation of the Crosswalker with chip-card function on the EN-Track software.

You must have at least version 6.5 of the EN-Track software to be able to operate the machine in a chip-controlled manner.

The patient shall receive instructions to stop training when prescribed environmental conditions are not met.

Technical changes reserved.

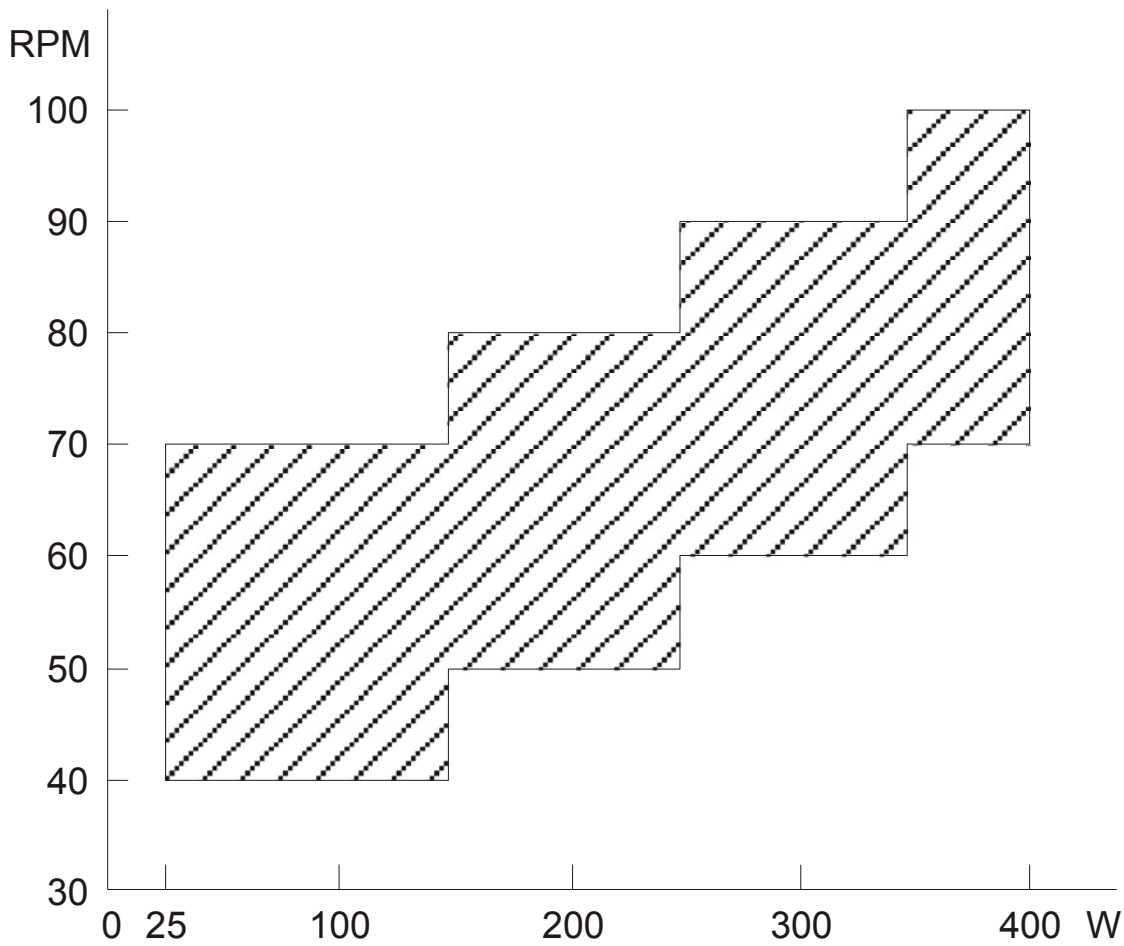
Fig. 20 Accuracy

---

## 17 Ordering details

---

3446668 Crosswalker

3446673 Crosswalker incl. hand/pulse measurement

The following accessories for the Crosswalker are available:

3496161 Bookstand

3496839 Polar® chest strap for heart rate monitoring

3496840 Elastic band for Polar® chest strap for heart rate monitoring

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