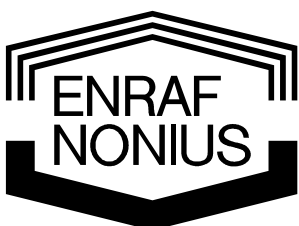
EN-Free

Bedieningshandleiding
Operating Instructions
Gebrauchsanweisung
Manual de uso

Copyright:

Enraf-Nonius B.V.
P.O. Box 12080
3004 GB Rotterdam
The Netherlands
Tel: +31 (0)10 – 20 30 600
Fax: +31 (0)15 – 25 61 699
info@enraf-nonius.nl
www.enraf-nonius.com

Part number: 1446750_43
3rd edition, January 2006

EN-Free

Bedieningshandleiding
Operating Instructions
Gebrauchsanweisung
Manual de uso

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CONTENT

1	Foreword	17
2	Safety aspects	17
3	Legal aspects	18
	3.1 Product liability.....	18
	3.2 Limitation of liability.....	18
4	Installation	18
	4.1 Installation and connection.....	18
5	Operation	19
	5.1 The machine.....	19
	5.2 EN-Free console.....	19
	5.3 The chip-card.....	19
	5.4 Description of a training on repetition.....	20
	5.5 Description of a training on Time.....	22
6	Maintenance	24
	6.1 Technical maintenance.....	24
	6.2 Cleaning.....	24
	6.3 Environmental information.....	24
7	Faults	24
	7.1 None of the displays lights up.....	24
	7.2 The exercise display shows: ----.....	24
	7.3 In case of other faults.....	24
8	Technical data	25

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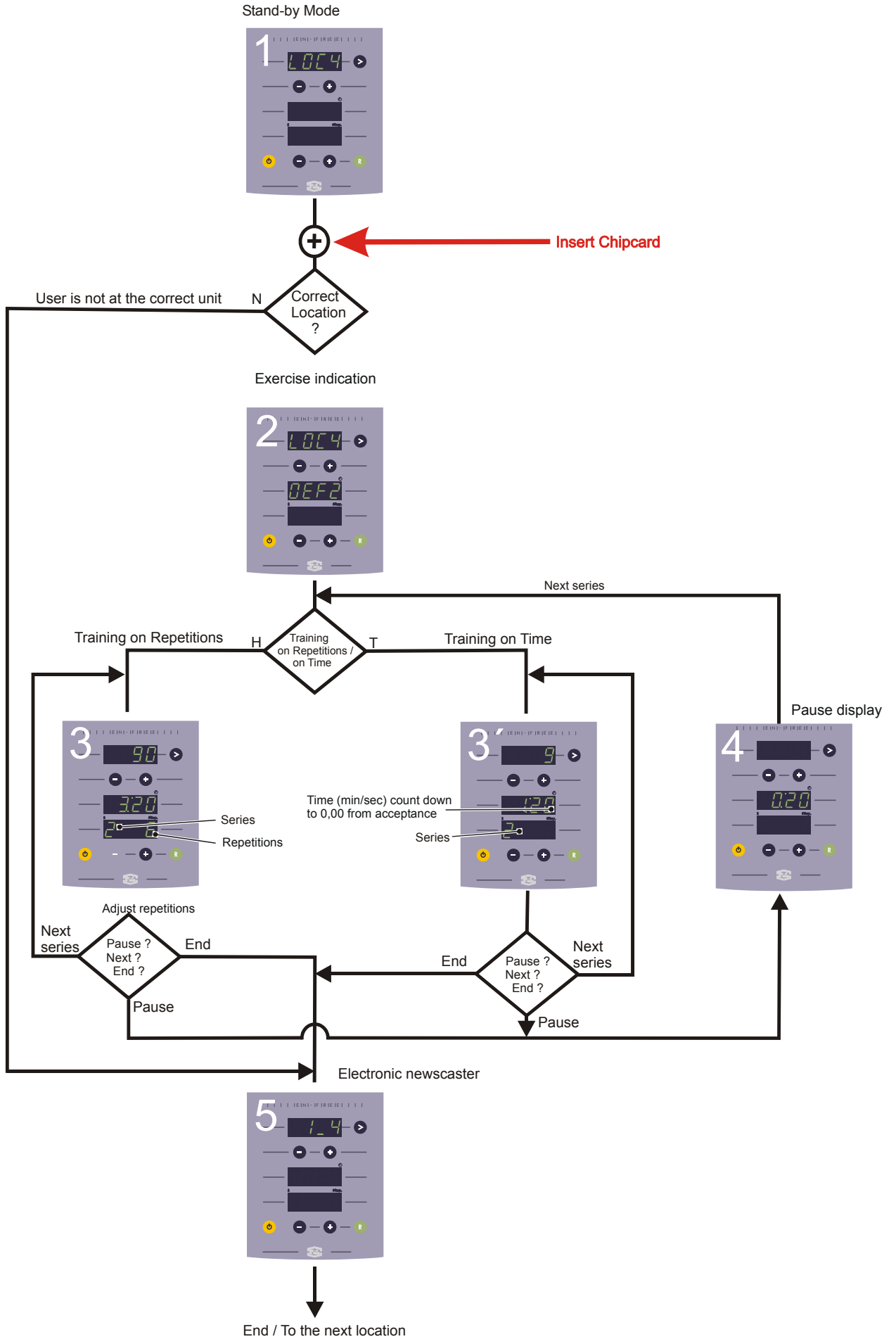
Work flow EN-Free

1 Foreword

Congratulations on choosing EN-Free registration equipment. Enraf-Nonius is sure that your purchase will give you many years of satisfaction.

This manual is intended to serve as an introduction to and an explanation of the EN-Free registration equipment. Read this manual through carefully before you start using your EN-Free. We recommend that you begin by immediately trying out on your EN-Free equipment the whole range of actions described in this manual.

To make proper use of your EN-Free registration equipment it is important that you read the instructions before use.

Before you ask anyone to train with the EN-Free registration equipment you must inform him/her about how the equipment is generally used and make sure that he or she is familiar with the operating instructions.

2 Safety aspects

Safe execution of the procedures in this manual requires technical experience and a general knowledge of safety precautions as well as a general knowledge of the human loco-motor system.

Warnings, Cautions and Notes have been used throughout the manual to bring special matters to the reader's immediate attention.

- A **Warning** concerns danger to the safety of the user or technician;
- A **Caution** concerns an act that might damage the equipment;
- A **Note** highlights a statement that deserves more emphasis than the general text, but which does not merit a **Warning** or a **Caution**.

It is important to follow the correct sequence of steps in a procedure to ensure safe handling of the equipment and the prevention of damage. Never, therefore, change the sequence of a procedure.

Warnings

- For safety reasons, always maintain a safe distance from a unit that is being used for training; you should always be at least 1 meter from the outer edges of a unit while it is in use.
- If pain is experienced during use stop the training immediately and seek professional medical help.

Caution

- EN-Free is powered by low-voltage current. Connecting an adapter other than that prescribed by Enraf-Nonius can adversely affect the safety of the user and the functioning of the unit, and is therefore not permitted.
- Arrange the electrical cabling so that people cannot trip over it
- Make sure that electrical cabling cannot become pinched or trapped between the moving parts and the unit itself.

Note

- If your EN-Free should malfunction, consult chapter 7 of this manual
- Repairs or maintenance performed by a service technician who is not authorized by Enraf-Nonius and/or the use of (spare) parts that were not originally made by Enraf-Nonius result in a direct loss of any guarantee.

3 Legal aspects

This manual and the information it contains are the property of Enraf-Nonius B.V. (Delft, The Netherlands). Enraf-Nonius B.V. disclaims any responsibility for personal injury and/or damage to equipment caused by:

- Negligence or disregard of a Warning or a Caution.
- Deviation from any of the described procedures.
- Performance of activities that are not described in this manual.
- Ignorance of the general safety precautions for handling tools and the use of electricity.
- Usage of the device in a manner other than that specified in this manual.

3.1 Product liability

A law on Product Liability has come into force in many countries. This Product Liability law implies, among other things, that once a period of 10 years has elapsed after a product has been brought into circulation, the manufacturer can no longer be held responsible for possible shortages of the product.

3.2 Limitation of liability

If permitted by the applicable legislation, Enraf-Nonius or its suppliers or sellers shall not be held liable under any circumstances for indirect, exceptional, incidental or consequential damage arising from the use of or the impossibility of using the product, including but not limited to damages resulting from loss of goodwill, work interruption, computer defects or faults, or any other commercial damages or losses, even if Enraf-Nonius or its suppliers or sellers were made aware of the possibility of such damage and irrespective of the legal or equity principle (contract, unlawful act or otherwise) on which the claim is based. Enraf-Nonius shall not, on the basis of the provisions of this agreement, be held liable for damages that amount to more than the sum that Enraf-Nonius has received from you for this product and any fees for support of the product that Enraf-Nonius has received on the basis of a separate support contract. In the event of death or bodily injury arising from the negligence of Enraf-Nonius this limitation does not apply if the applicable legislation forbids such a limitation.

Enraf-Nonius is not liable for the consequences of the incorrect information and advices of its staff or errors appearing in this manual and/or any other accompanying documentation (including commercial documents).

The other party (the user or user's representative) is obliged to hold Enraf-Nonius harmless against all claims of third parties, whatever they may be called and on whatsoever grounds they may be based.

4 Installation

To mount the control unit on the pedestal or wall bracket, please consult the enclosed instruction sheet.

4.1 Installation and connection

1. Mount the unit on the pedestal or wall bracket before connecting it to the electricity supply.
2. Insert the connector of the adapter into the connector on the power cable of the EN-Free control unit.
3. Arrange the power cable so that there is no danger of anyone tripping over it. Make sure that the cable cannot become pinched or trapped.
4. Plug the adapter into a wall socket with a mains voltage that corresponds to that for which the adapter is designed.
5. EN-Free is now ready for use.

5 Operation

All the items of training equipment in the EN-Strength range are operated in the same way. The operating panel of the EN-Free is shown on the folding page.

5.1 The machine

- [1] Attachment point to pedestal or all bracket
- [2] Chip-card reader

5.2 EN-Free console

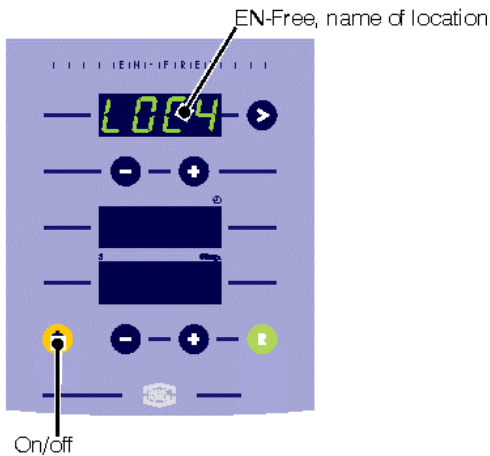
- [3] Orange ON/OFF button.
 - [4] Display that shows the exercise currently being performed. The exercises are explained in the exercise booklet. During the set-up and training this display shows the pre-set resistance level, the number of repetitions and the time. This display also shows the duration of the training, alternating with the duration of the pause between Series. The duration of the training is only displayed when this parameter is selected instead of Number of Repetitions.
 - [5] “-” key for decreasing the target parameter*. (This can be blank, e.g. for a push-up exercise).
 - [6] “+” key for increasing the target parameter.
 - [7] Display for location: If a programmed chip-card with more training units is being used and the last series for this unit is finished, this display will show the next sequence number or -PC-.
 - [8] Key for accepting, starting and ending an exercise.
 - [9] Display for “S” (number of Series).
 - [10] Display for “#Rep.” Unit of Repetitions.
 - [11] “R” key: Pulsing this key will end the current Series and reset the Repetition counter to “0”. If further Series are programmed, the training unit software will enter the Pause phase before the next Series begins. If the Reset key is pressed during the last Series for this unit, the display of the resistance level will show the sequence number of the next training unit on the chip-card. If the “R” key is pressed when all the entered sessions have finished, the resistance level display will show “-PC-”, indicating that you must take the chip-card back to your PC.
 - [12] “+” key will increase the number of repetitions during the set-up phase.
 - [13] “-” key will decrease the number of repetitions during the set-up phase.
- * target parameter refers to (for example) the number of kg while performing a dumbbell exercise (It can be blank, e.g. for a push-up exercise).

5.3 The chip-card

- [14] Chip-card LED: If the LED is on, this indicates that the chip-card is busy performing a read or write operation to/from the card.

5.4 Description of a training on repetition


Stand-by Mode

1. Stand-by mode: indicates the location of this EN-Free


Insert the chip-card into the EN-Free (with the chip pointing towards you)

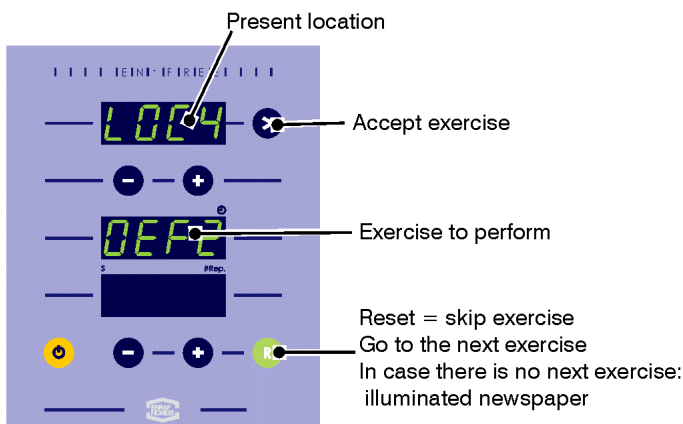Exercise indication


1. Exercise indication
2. Accept the exercise by pressing >

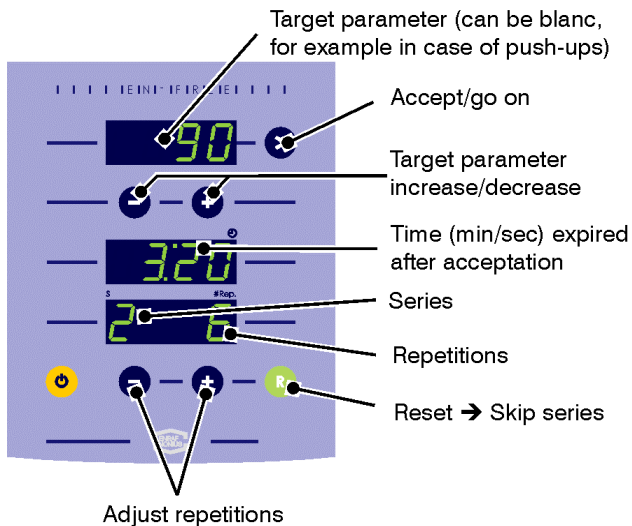Training on repetitions, 1^e Series


1. Change parameter if necessary (90) using + and -
2. Change repetitions if necessary using + and -
3. Press > to start and complete the first series (2) with 6 repetitions (time starts running from 0:00)
4. Press > after finishing the first series (2)

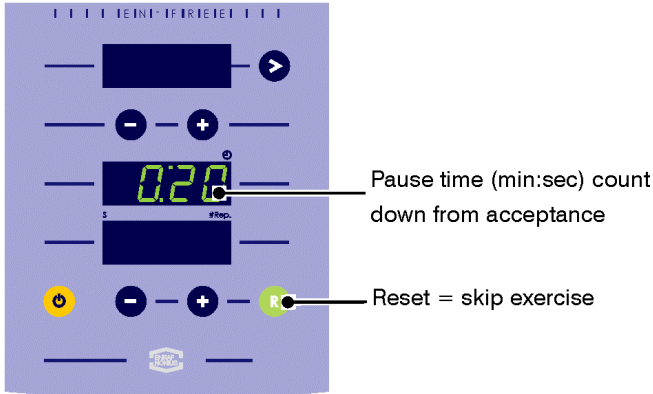Pause display


1. The pause (if selected) runs from 0:20 to 0:00

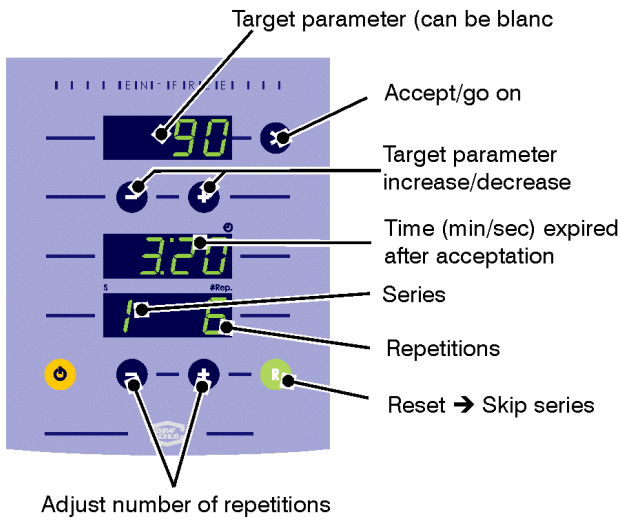Training on repetition, 2° Series

1. Press > to start and complete the 2nd series (1) with 6 repetitions (time starts running from 0:00)
2. Press > after finishing the 2nd series

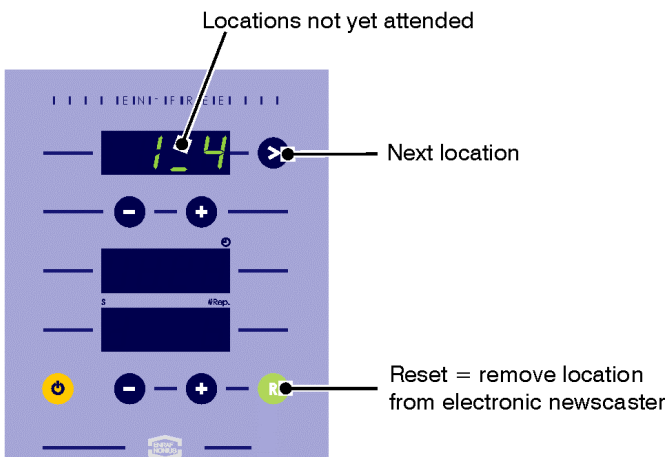Electronic newscaster


If a programmed chip-card with more training units is being used and the last series for this unit is finished, this display will show the next training units.

When all the entered sessions have finished, the resistance level display will show *PC*, indicating that you must now take the chip-card back to your personal computer.

5.5 Description of a training on Time

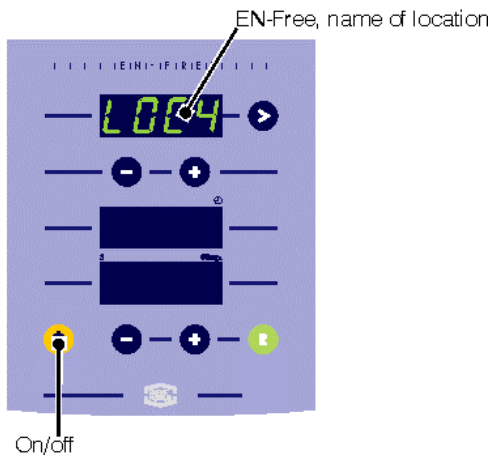
Stand-by Mode

1. Stand-by mode: indicates the location of this EN-Free


Insert the chip-card into the EN-Free (with the chip pointing towards you)

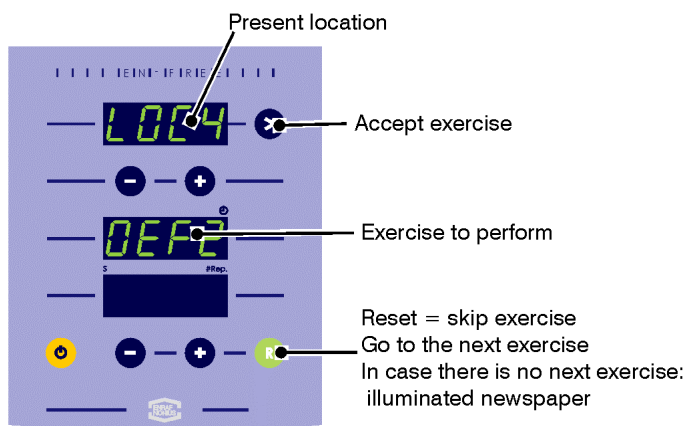Exercise indication


1. Exercise indication
2. Accept the exercise by pressing >

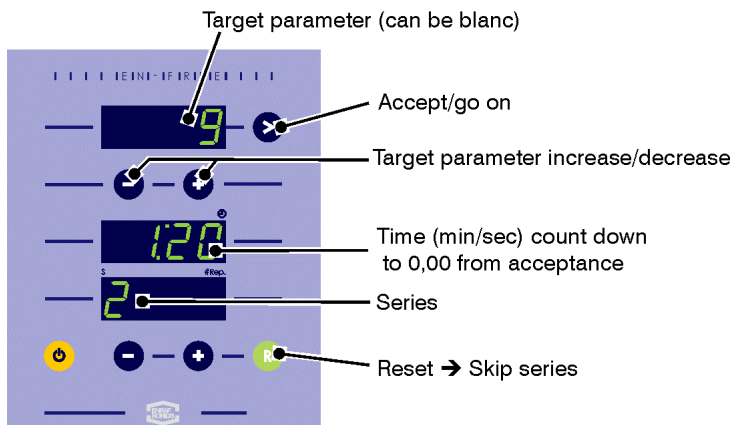Training on time, 1° Serie


1. Change parameter (9 kg) if necessary (9) using + and -
2. Press > to start and complete the first series (2), count down from 1:20 to 0:00. A beep follows and 0:00 blinks on the display.
3. Press > after finishing the series

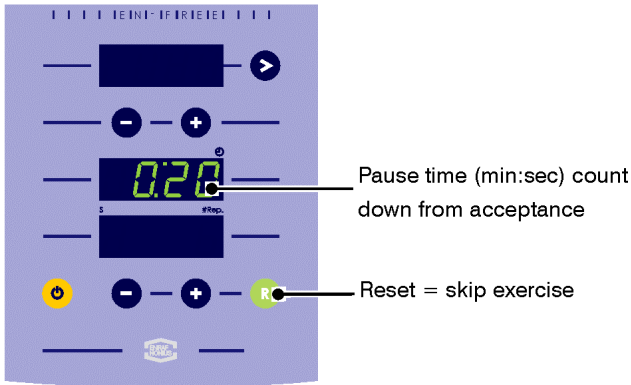Pause display


1. The pause (if selected) runs from 0:20 to 0:00

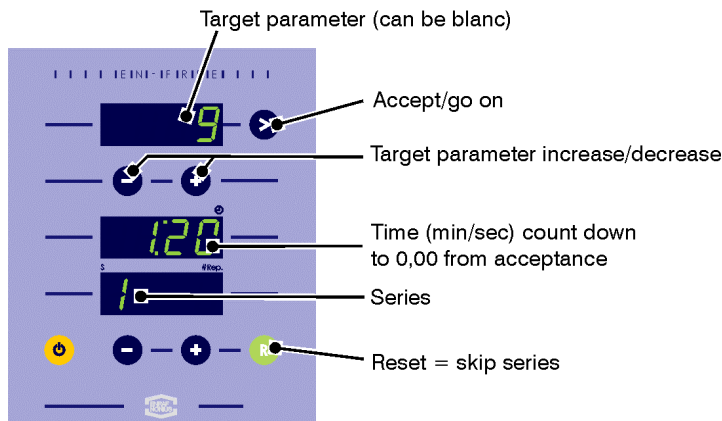Training on time, 2° Serie

1. Press > to start and complete the second series (1). Time starts running from 0:00
2. Press > after finishing the second series (1)

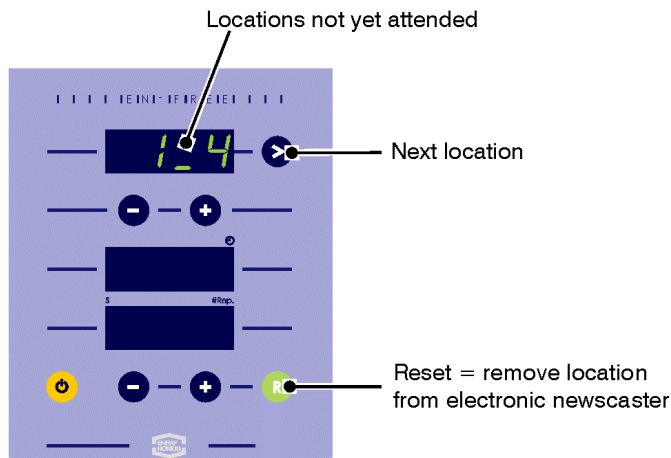Electronic newscaster

If a programmed chip-card with more training units is being used and the last series for this unit is finished, this display will show the next training unit(s).

When all the entered sessions have finished, the resistance level display will show 'PC', indicating that you must now take the chip-card back to your personal computer.

6 Maintenance

6.1 Technical maintenance

Inspection and/or maintenance must be performed in accordance with the procedures described in the service manual for the machine. It is also recommended that a file be maintained in which all the maintenance activities are documented. In some countries this is actually a legal requirement.

Maintenance and repair may only be performed by an authorized body. Enraf-Nonius is not liable for the consequences of any maintenance or repair performed by unauthorized persons.

It is not permitted for anyone other than authorized persons to open the machine, and doing so invalidates the warranty.

6.2 Cleaning

Disconnect the EN-Free adapter from the power source before cleaning. The exterior of the EN-Free can be cleaned with a soft cloth dampened with a non-aggressive cleaning liquid.

6.3 Environmental information

Your EN-Free contains materials that can be recycled and/or are noxious to the environment. At the end of the service life, specialized firms can dismantle the EN-Free and sort the noxious materials from the materials for recycling. By following this procedure you contribute to a better environment.

Please ensure that you are well informed about the local rules and regulations regarding the disposal of equipment and accessories.

7 Faults

7.1 None of the displays lights up

If the machine is switched on but none of the displays lights up, the EN-Free is probably not receiving any current. Check that the plug of the mains voltage adapter is properly seated in the socket. It's also possible that the adapter is defective. Exclude this possibility by trying the EN-Free with an adapter from another (properly functioning) EN-Free. If the machine is now working then replace the defective adapter. If the machine still doesn't work it may be that the mains voltage has failed on the wall socket in question. If this is not the case then contact your EN-Free supplier or Enraf-Nonius B.V. (Delft, The Netherlands.) Repairs and installations should only be performed by service technicians authorized by Enraf-Nonius.

7.2 The exercise display shows: ----

Four dashes appear in the window for indicating the exercise and in the window for the series and repetitions. This indicates that the chip-card has been inserted incorrectly. Insert the card properly into the card reader / writer.

7.3 In case of other faults

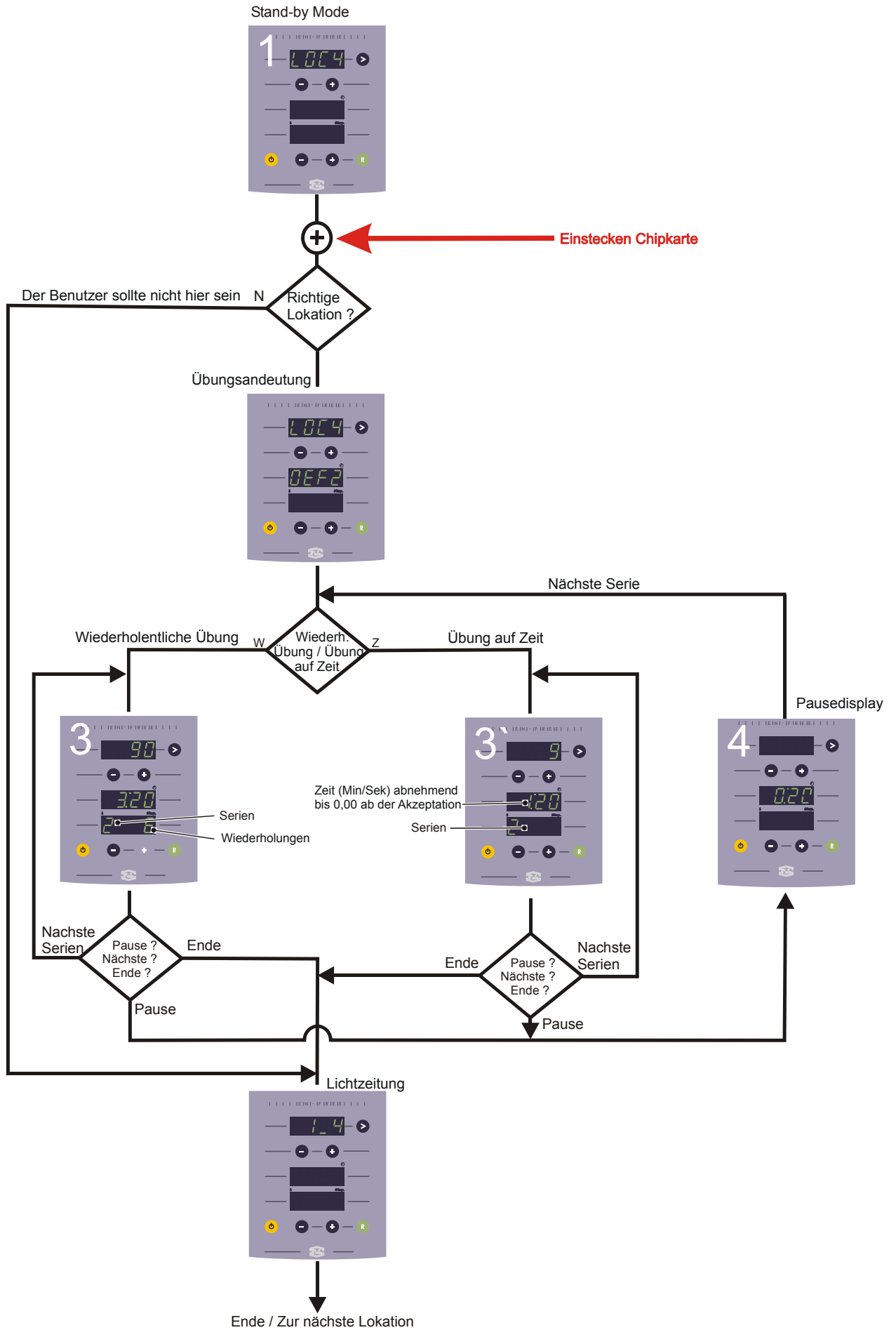
In case of other faults with the EN-Free equipment please contact your supplier of the EN-Free equipment. The service must be carried out by authorised Enraf-Nonius service personnel.

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