ManuXelect

Bedieningshandleiding
Operating Instructions
Gebrauchsanweisung
Mode d'emploi
Instrucciones de manejo
Guía de usuario

Copyright:

Enraf-Nonius B.V.
P.O. Box 12080
3004 GB Rotterdam
The Netherlands
Tel: +31 (0)10 – 20 30 600
Fax: +31 (0)10 – 20 30 699
info@enraf-nonius.nl
www.enraf-nonius.com

Part number: 1550750-41
April 2010

ManuXelect

Bedieningshandleiding
Operating Instructions
Gebrauchsanweisung
Mode d'emploi
Instrucciones de manejo
Guía de usuario

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7

1 Foreword

We are pleased that you have purchased a ManuXelect treatment couch. This examination and treatment couch is specially designed for the demands of the physical and manual therapist. The anatomically segmented top part combined with the wide array of section positions enable you to position your patient virtually however you wish.

During development and production of this couch a great deal of care was put into the reliability, safety, ease of use and durability. It is important to read the following instructions carefully before using your ManuXelect treatment couch properly and safely.

This operating instructions deals with:

- ManuXelect 2-section, head section
- ManuXelect 2-section, back section
- ManuXelect 3-section
- ManuXelect accessories

2 Product liability

A law on Product Liability has become effective in many countries. This Product Liability law implies, amongst other things, that once a period of 10 years has elapsed after a product has been brought into circulation, the manufacturer can no longer be held responsible for possible shortcomings of the product.

To the maximum extent permitted by applicable law, in no event will Enraf-Nonius or its suppliers or resellers be liable for any indirect, special, incidental or consequential damages arising from the use of or inability to use the product, including, without limitation, damages for loss of goodwill, work and productivity, computer failure or malfunction, or any and all other commercial damages or losses, even if advised of the possibility thereof, and regardless of the legal or equitable theory (contract, tort or otherwise) upon which the claim is based. In any case, Enraf-Nonius's entire liability under any provision of this agreement shall not exceed in the aggregate the sum of the fees paid for this product and fees for support of the product received by Enraf-Nonius under a separate support agreement (if any), with the exception of death or personal injury caused by the negligence of Enraf-Nonius to the extent applicable law prohibits the limitation of damages in such cases.

The opposing party (product's user or its representative) shall disclaim Enraf-Nonius from all claims arising from third parties, whatever nature or whatever relationship to the opposing party.

3 Installation

1. Take the ManuXelect and any extra parts out of the packaging and check everything for possible transport damage.
2. Read this manual carefully before using the equipment.
3. Mains supply connections must comply with the national requirements regarding medical rooms.
4. Prior to connection of this apparatus to the mains supply, check that the voltage and frequency stated on the type plate correspond with the available mains supply.
5. Avoid exposure to direct sunlight, UV radiation, rain, excessive dust, moisture, mechanical vibrations and shocks.
6. The ManuXelect couches are not intended for use in so-called "wet rooms" (hydrotherapy rooms).
7. The apparatus may not be used in close proximity (i.e. less than 2 metres) to shortwave or microwave equipment.
8. The ManuXelect should be placed on a flat surface.
9. Turn the adjustable foot until it touches the ground and the couch is standing stable.
10. For correct assembly and safe use of possible accessories one must consult the separately provided document (0167291 [Gynaecological set] and 0167282 [Paper roll holder]).

4 General instructions

1. Never adjust a section when it supports the full weight of the patient. Each section of this ManuXelect is supported by gas springs. As well as easing operation they offer full support of the sections. However, these gas springs are not strong enough to lift a section while it supports the full weight of the patient.
2. Use both hands to change the couch settings: one hand for unlocking and the other to adjust the section.
3. A maximum patient's weight of **150 kg** is allowed with the **6000N motor KL95** (standard motor) when the height of the couch is being adjusted;
A maximum patient's weight of **250 kg** is allowed with the **9000N motor SL80** (optional motor) when the height of the couch is being adjusted.
4. Never let the motor exceed a few minutes continuous operation. If this should happen, a thermal cut out shuts off the power and protects the motor against overheating. In that case, let the motor cool down until it functions again.
5. The couch is not anti-static.
6. Always use the original Enraf-Nonius accessories that are meant for this product.
7. The apparatus is only meant to be used by competent personnel in physiotherapy, rehabilitation or adjacent areas. Avoid any use of the equipment by unauthorized personell.
8. When used by children, supervision by a knowledgeable (para)medic is obligatory. At all times keep children away from the equipment when no supervision is at hand or use a safety switch.
9. Use a clean couch cover / towel / terry cloth / paper cover for every individual. This is required to guarantee a hygiënic use.

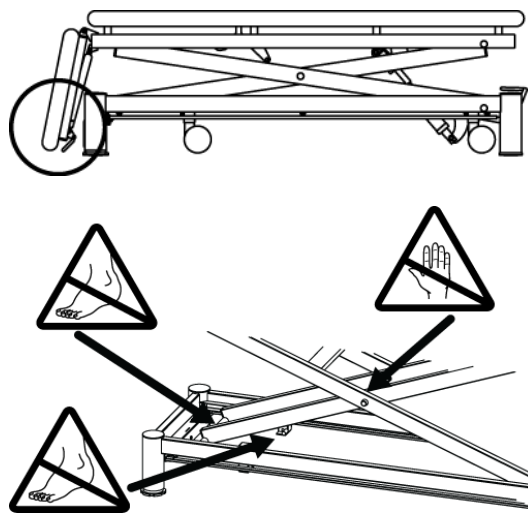
5 Safety instructions

5.1 Height adjustment (general)

Ensure that neither the patient nor the operator can get any part of the body or clothes near or in between the moving parts of the mechanism when the couch is being raised or lowered.

Be particularly careful for the situation indicated below (the couch is set in the lowest possible position):


Underneath the head section, between the head section and floor.

5.2 Height-adjustment

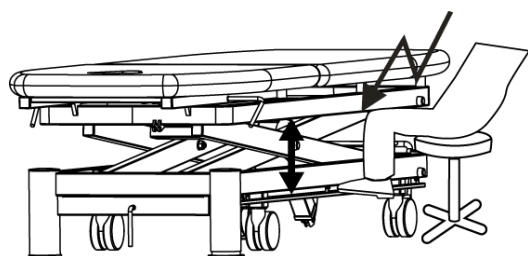
Ensure that you will never rest your feet on the all-around-operation. Especially when seated next to the couch it can lead to a dangerous situation in which your leg can be jammed between the section and the all-around-operation.

5.3 Danger of tilting

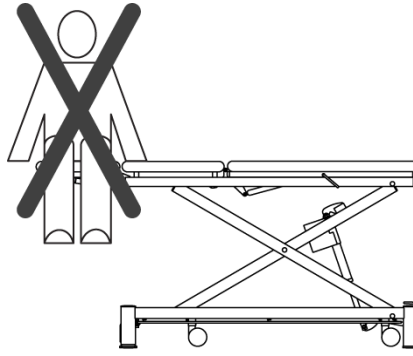We advise against sitting on the head and leg section of the couch.

5.4 Load

The Head section is not suited to sit upon and must be used to support the head only.

5.5 Danger of squeezing the extremities

On this couch warning-stickers are placed.

Warning: When adjusting this couch, keep hands and feet away from the couch for at least 20 cm, especially on places that has been marked by these stickers.

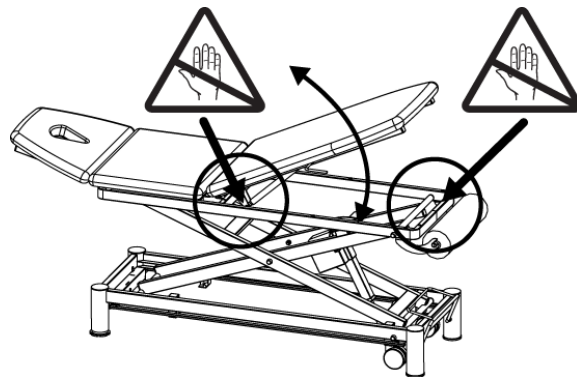
5.6 Adjustment of the sections

When adjusting the sections, squeezing-danger can arise at contact points of the section.

Warning: keep enough distance between you or your clients and these points.

6 Cleaning, maintenance and malfunctions

The couch and accessories may be cleaned with a damp cloth and a domestic liquid cleaner. Use lukewarm water and a non-abrasive liquid household cleaner (no abrasive, no alcohol content solution).

6.1 Cleaning

Tested desinfectants

Manufacture	Name of desinfectant	Upholstery AKV Peri
JohnsonDiversey	McDonald's SK	<input checked="" type="checkbox"/>
	McDonald's HA	<input checked="" type="checkbox"/>
	MC Playlandreiniger Taski force Spray Z22	<input checked="" type="checkbox"/>
	Sprint Performer	<input checked="" type="checkbox"/>
	Mc Donald's UR conc	<input checked="" type="checkbox"/>
	Sprint Multiuso	<input checked="" type="checkbox"/>
	Suma Quicksan D4.3	<input checked="" type="checkbox"/>
	Sprint Spitfire Spray	<input checked="" type="checkbox"/>
B. Braun Melsungen AG	Helipur	<input checked="" type="checkbox"/>
	Meliseptol®	<input checked="" type="checkbox"/>
	Meliseptol® rapid	<input checked="" type="checkbox"/>
	Melisep SF®	<input checked="" type="checkbox"/>
CosiMed GmbH	CosiMed Desinfektionsmittel	<input checked="" type="checkbox"/>
Schülke & Mayr GmbH	ESEMFIX	<input checked="" type="checkbox"/>
	Mikrozid AF	<input checked="" type="checkbox"/>
Henkel Hygiene GmbH	Incidur Spray	<input checked="" type="checkbox"/>
	Incidur N	<input checked="" type="checkbox"/>
	Incidin perfekt Konzentrat	<input checked="" type="checkbox"/>
	Incidin Plus	<input checked="" type="checkbox"/>
	Incidin Extra N	<input checked="" type="checkbox"/>
	Incidin Extra	<input checked="" type="checkbox"/>
Peppler GmbH & Co. KG	Incidin liquid Spray	<input checked="" type="checkbox"/>
	Micro Clean	<input checked="" type="checkbox"/>
Reckitt Benckiser	Sagrotan Hygienespray	<input checked="" type="checkbox"/>

Result:

The tested disinfectant does not cause any damage at the surface of the upholstery vinyl.

Comment:

- The disinfectant should not be changed
- Only use one disinfectant and rinse with clear water. Never use a mixture of various disinfectant or cleaners!
- As we have no influence on the production and raw materials of the disinfectants, we can give no general guarantee of the resistance of disinfectants.

6.2 Maintenance

6.2.1 Maintenance operation handles

- Check the operation of every section adjustment regularly (at least once every month). For correct operation keep the bars of the mechanism clean and free of grease.
- Check at least once every 6 months the functioning of the height adjustment if this has not been used for a long time. Disconnect the mains cable from the electricity supply if the couch is not being used for a long period.

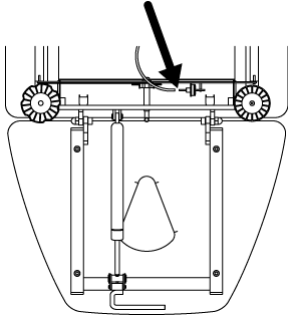
6.2.2 Maintenance advice

The couch should be checked once a year by authorized personnel of Enraf-Nonius or by its authorized distributor.

6.3 Malfunctions

Disconnect the mains cable from the electricity supply if the all-around-operation is not working appropriately.

Then check whether the three operation elements of the all-around-operation are in a neutral position: the bar on the left and on the right plus the handle

At the short sided edge of the couch, where you can find the small operation handle, there is a cord that runs to the underside of the couch. Try to follow this cord with your hand as far as possible, till you can pull it loose carefully from the small pin. – see the small illustration alongside, illustration is a bottom view –. Uncouple the cord for a moment and put it back again. Afterwards you can connect the mains cable with the electricity supply and bring the all-around-operation back into use.

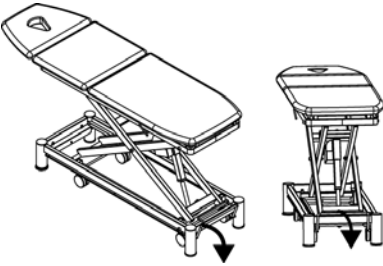
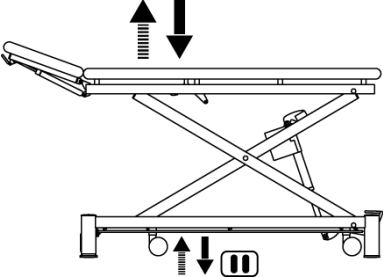
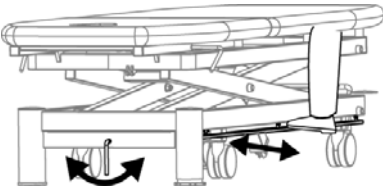
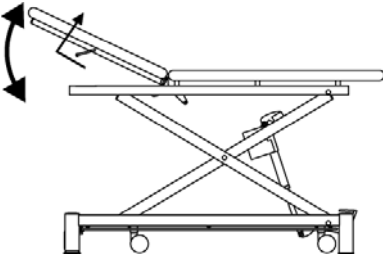
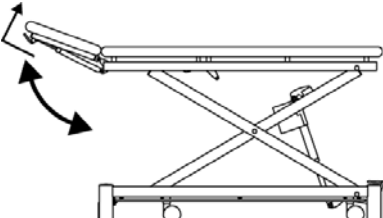
6.4 Environment Information

Your ManuXelect contains materials which can be recycled and/or are noxious for the environment. At the end of the duration of life, specialised concerns can take apart the ManuXelect and sort out the noxious materials and materials for recycling. By doing so you contribute to a better environment.

Please ensure that you are well informed of the local rules and regulations regarding to the removal of equipment and accessories.

7 Operation

The operation of your ManuXelect treatment couch is simple and safe. The illustrations show how the various adjustments are done

<p>7.1 Mobile stand</p> <p>The couch is supplied with wheels for transfer. The wheels appear by pressing the pedal. When in position, retract the wheels to give full stability before treating the patient (optional).</p>	
<p>7.2 Electric height adjustment</p> <p>By pressing the foot switch continuously the required height can be reached.</p>	
<p>7.3 Height adjustment with the all around operation (optional)</p> <p>By pressing or pulling the all around operation - which is located underneath the framework - with your foot, the required height of the couch can be set. At the head section of the couch, the height can be adjusted by turning the handle to the left or to the right.</p> <p>Att. Never let your full weight rest on the switchbar!</p>	
<p>7.4 Adjusting the back section</p> <p>Use one hand to pull the handle underneath the back section, while adjusting the section in the desired position. Release the handle and the section will be fixed.</p>	
<p>7.5 Adjusting the head section</p> <p>Use one hand to pull the handle underneath the head section while adjusting the section in the desired position. Release the handle and the section will be fixed</p>	

8 Technical data

Height adjustment	:	from 48 till 105 cm
Length	:	ca. 207 cm
Width	:	ca. 67 cm
Lifting capacity	:	150 kg with the standard motor KL95 of 6000N
	:	250 kg with the optional motor SL80 of 8000N
Motor speed	:	6 mm/sec for both types
Power supply	:	230V~ 50/60 Hz or 115V~ 50/60 Hz
Current consumption	:	1,5A for the 6000N motor
	:	2,0A for the 8000N motor
Moisture resistance	:	according IP 55
Medical device classification	:	I
Safety class (IEC 60601-1)	:	II (double insulated).
Upholstery	:	Type AKV Peri
Fire safety	:	DIN EN 1021 part 1/2
Free of phthalates	:	
Biological	:	DIN EN ISO 10993-5, 1999-11
	:	DIN EN ISO 10993-10, 1999-12

Technical modifications reserved.

9 Ordering data

For the ordering data of the ManuXelect, standard accessories and additional accessories we refer to the Catalogue for Physiotherapy or to www.enraf-nonius.com.

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